

DATASHEET

Ambu® Mark IV resuscitators are reusable resuscitators designed for manual ventilation of adult, paediatric and infant patients. They are available in two sizes; Baby - suitable for patients with a body weight up to 20 kg, and Adult - suitable for patients with a body weight of more than 15 kg.


Featuring a single-shutter valve system, the Ambu® Mark IV has become an established choice of manual resuscitator, providing reliable ventilation to patients in both emergency and in-hospital environments for over 15 years.

The patient valve is housed in a transparent housing, which allows for users to observe the operation of the valve and ensure it's functionality. In addition, there is a swivel connector located between the patient valve and face mask that allows for 360° rotation of the resuscitator relative to the patient, this means that the operator is able to change position without interrupting ventilation.

On the Ambu® Mark IV Size Baby resuscitator, a built-in pressure limiting system is included to reduce the risk of barotrauma. If higher pressures are required, the override cap can simply be closed to prevent activation.

After use, the Ambu® Mark IV resuscitators are capable of being sterilised at a temperature of 134 °C/273.2 °F (not applicable for oxygen reservoir tube and size 0A face mask) and can be reprocessed 30 times (except the oxygen reservoir bag which can be reprocessed 15 times).

Ambu® Mark IV Resuscitators

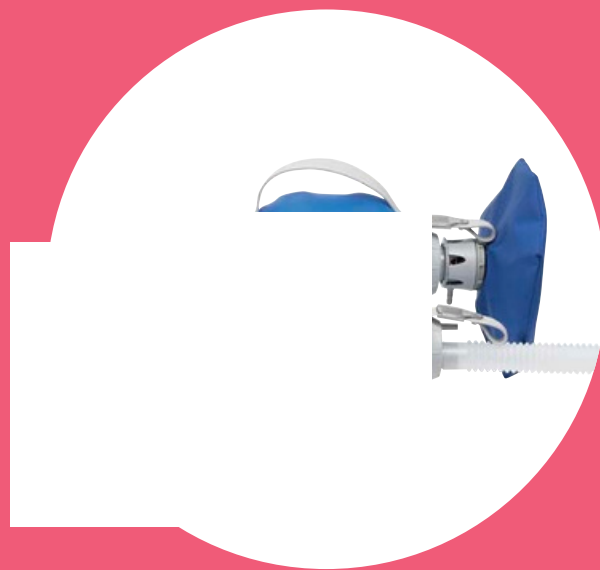

KEY BENEFITS

Swivel connection allows for 360° rotation, making it possible to change position without interrupting patient ventilation

Single-shutter valve system for reliable functionality

Pre-attached splash guard diverts fluids and expired air away from the user, for additional protection

The Ambu® Mark IV can be sterilised at a temperature of 134 °C/273.2 °F (except oxygen tube reservoir and size 0A face mask)

Integrated pressure-limiting valve with override cap on Ambu® Mark IV Size Baby for patient safety

Ambu

SPECIFICATIONS

	Mark IV Size Adult	Mark IV Size Baby
Resuscitator volume	Approx. 1450 ml	Approx. 420 ml
Delivered volume one hand	600 ml 900 ml Approx.	300 ml
Delivered volume two hands	270 x 130 mm Approx.	-
Dimensions (length x diameter)	415 g	Approx. 265 x 80 mm
Weight of the resuscitator (without oxygen reservoir and mask)		Approx. 190 g
Patient connector	Outside 22 mm male (EN ISO 5356-1)	
Expiration connector (for PEEP valve attachment)	Inside 15 mm female (EN ISO 5356-1)	
Manometer port connector	30 mm male (EN ISO 5356-1)	
Bag refill valve connector	Ø 4.2 +/- 0.1 mm	
	Inside 32 mm female	-
O2 Inlet Connector	According to EN ISO 13544-2	
Dead space	≤ 5 ml + 10 % of the delivered volume	≤ 5 ml + 10 % of the delivered volume
Leaks (both ways)	Non-measurable	Non-measurable
Pressure-limitation valve	-	40 cmH2O (4.0 kPa)
Oxygen reservoir bag volume	Approx. 1500 ml	Approx. 1500 ml (bag) Approx. 100 ml (tube)
Operation temperature	-18 °C to +50 °C (0 °F to +122 °F)	
Storage temperature	-40 °C to +60 °C (-40 °F to +140 °F)	

The Ambu Mark IV resuscitators comply with EN ISO 10651-4:2002.

1
v

1



Integrated pressure-limiting valve with override cap on Ambu® Mark IV Size Baby for patient safety

ORDERING SPECIFICATIONS

Item no.	Description
Mark IV Size Baby	
299 006 000 299	With reservoir bag and silicone face mask size 0A
007 000 299 008	With reservoir bag and silicone face mask size 0
000 299 009 000	With reservoir bag and silicone face masks size 0 & 0A
299 001 000 299	With reservoir bag
003 000 299 004	With reservoir tube and silicone face mask size 0A
000 Mark IV Size	With reservoir tube and silicone face mask size 0
Adult 304004000	With reservoir tube
304002000	
304009000	With reservoir bag With reservoir bag and silicone
304010000	face mask size 5 With reservoir bag and silicone face
3040011000	masks size 2 & 5 With reservoir bag and silicone
	face masks size 0 & 2 With reservoir bag and
	silicone face masks size 2 & 5

MATERIAL

Product	Material
Inner bag	Silicone rubber
Outer cover	Silicone rubber
Patient valve housing	PSU PSU
Patient connector	PSU/stainless steel
Pressure-limiting valve*	Silicone rubber PP
Override cap*	TPV POM POM PE
Expiration connector	Silicone rubber
Cap for manometer port*	Silicone rubber
Inlet valve housing	Aluminium PSU
Screw cap (two parts)	
Reservoir tube*	
Valve discs	
Strap	
Nameplate	
Window with snap fastener	
Connector	
Oxygen reservoir bag***	POM
Adapter for reservoir bag*/***	PE/PUR
Reservoir housing incl. cap and screw cap***	Silicone rubber
Splash guard**/***	POM/PP
	PSU

* Mark IV Size Baby only
** Mark IV Size Adult only
*** Optional accessory

CE 2797 US: Rx only

Ambu

Ambu A/S
Baltorpbakken 13
2750 Ballerup
Denmark
T +45 72 25 20 00
ambu.com