


Wie man eine Lizenz für die AT-Report Software erhält

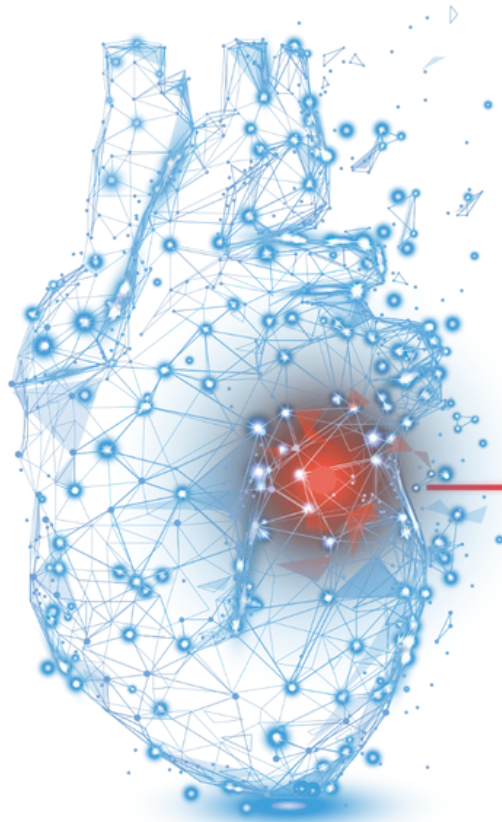

Wie man eine Lizenz für AT-Report S/W 1(DE) erhält

Confidential

Das Registrierungsformular wird angezeigt, wenn das

[Abbildung 1.15] Symbol in Lizenzbereich angeklickt wird, wie in

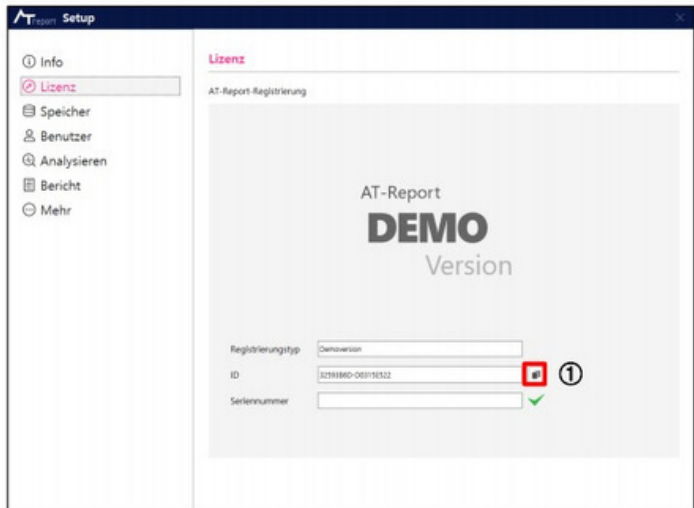


Abbildung 1.15 Lizenzierungsbereich

Wenn der Benutzer nach der Eingabe der Informationen auf das Symbol ② in Zwischenablage kopieren in [Abbildung 1.16] klickt, wird der Inhalt kopiert. Die kopierten Informationen sind an senden und alle einggegebenen Benutzerinformationen werden gesendet.


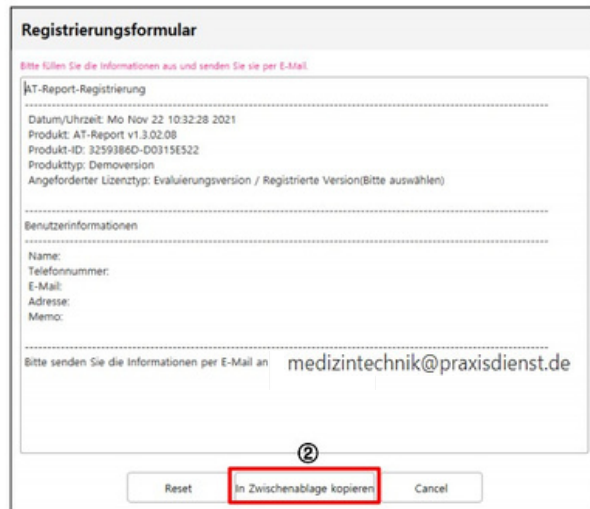


Abbildung 1.16 Zwischenablage-Symbol zum Kopieren des Anmeldeformulars und der ID

Wie man eine Lizenz für AT-Report S/W 2(DE) erhält

Confidential

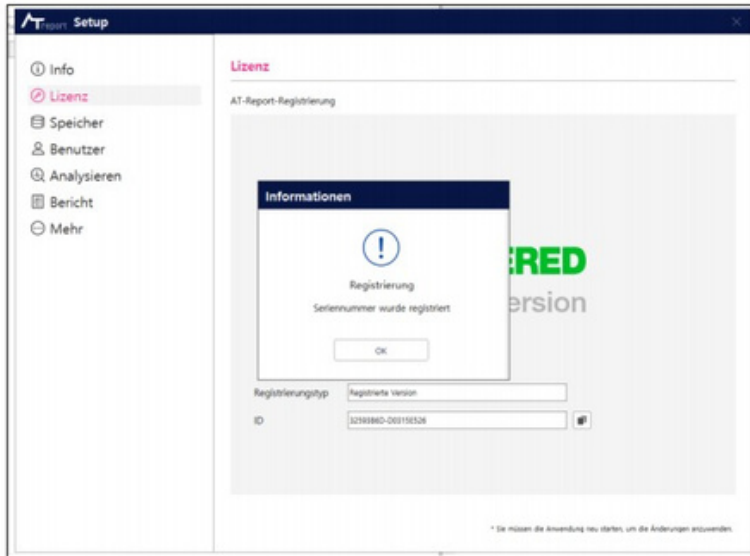

Nach Erhalt der Seriennummer, die einer von ATsens Co., Ltd. (medizintechnik@praxisdienst.de) ausgestellten ID entspricht, geben Sie die Seriennummer in das Feld Seriennummer unter dem Feld ID ein und klicken Sie dann auf das Symbol Bestätigen (✓), wie in [Abbildung 1.15] gezeigt.

Wenn eine korrekte Seriennummer eingegeben wurde, werden ein entsprechendes Symbol und eine Meldung angezeigt, die auf dem Typ der Seriennummer basieren, wie in [Abbildung 1.17] dargestellt.

Abbildung 1.17 Bildschirm des Eingabeergebnisses für die Seriennummer der Registrierungsversion

How to get license for AT-Report S/W 1(EN)

Confidential

The RegistrationFormwindowisgeneratedwhenthe

Registration icon in the License section is clicked, as shown in ①

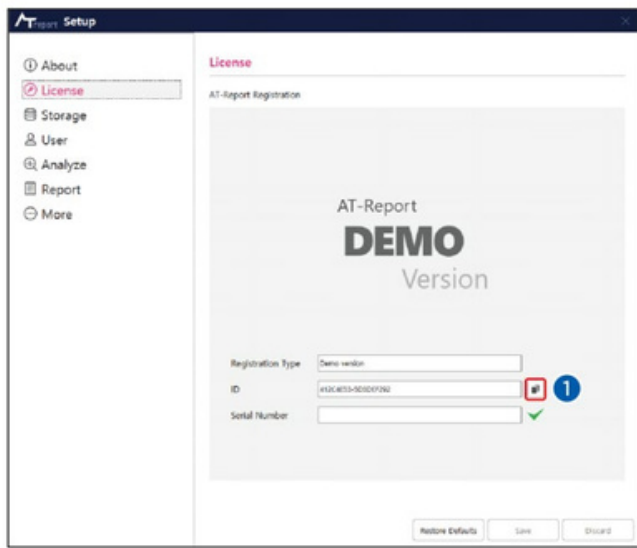


Figure 1.15 Setup License section

Once the user clicks the ② Copy to Clipboard icon in [Figure 1.16] after entering the information, the content is copied. The copied information is to be sent to ATsens Co., Ltd. (medizintechnik@praxisdienst.de) via email, and all of the entered user information is sent.



Figure 1.16 Clipboard icon for copying the Registration Form and ID

How to get license for AT-Report S/W 2(EN)

Confidential

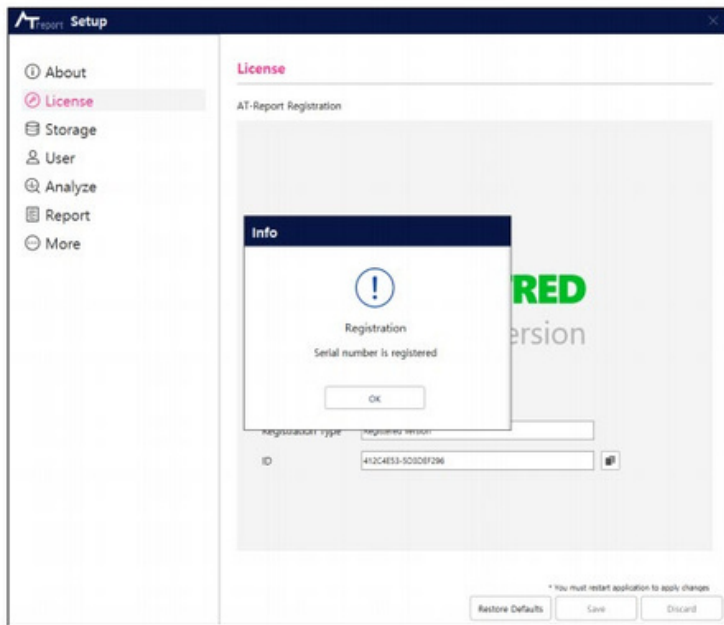


Figure 1.17 Registration version serial number input result screen

Upon receiving the serial number, which is equivalent to an ID issued by ATsens Co., Ltd. (medizintechnik@praxisdienst.de), enter the serial number in the Serial Number field located below the ID (✓) field and then click the Confirm icon (✓) as shown in [Figure 1.15]. If a correct serial number is entered, a corresponding icon and message will be displayed, based on the serial number type as shown in [Figure 1.17].

Thank you!

NEW FRONTIER
OF HEALTHCARE PLATFORM