


Wie man eine Lizenz für die AT-Report Software erhält

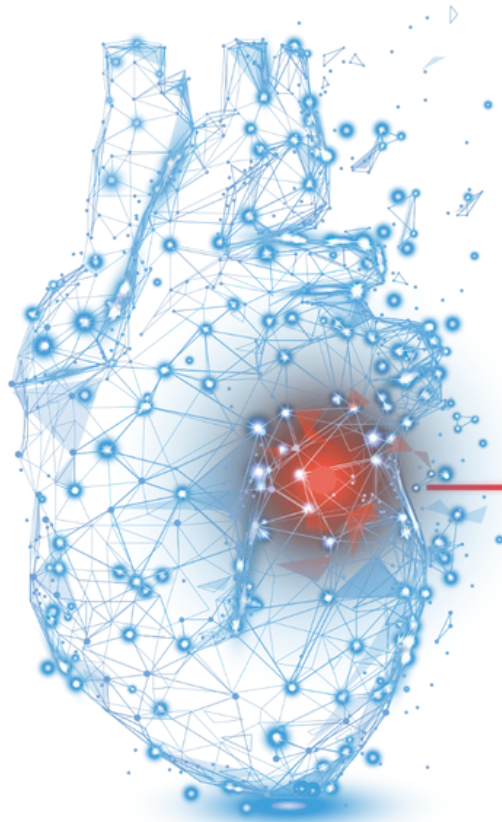

Wie man eine Lizenz für AT-Report S/W 1(DE) erhält

Confidential

Das Registrierungsformular wird angezeigt, wenn das

[Abbildung 1.15] Symbol ^① im Lizenzbereich angeklickt wird, wie in

Wenn der Benutzer nach der Eingabe der Informationen auf das

Symbol ^② In Zwischenablage kopieren in [Abbildung 1.16] klickt, wird der Inhalt kopiert. Die kopierten Informationen sind an senden und alle einggegebenen Benutzerinformationen werden gesendet.

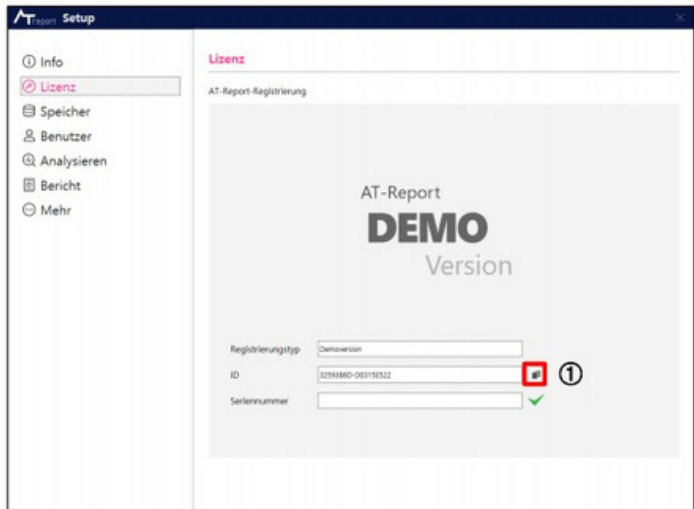


Abbildung 1.15 Lizenzierungsbereich


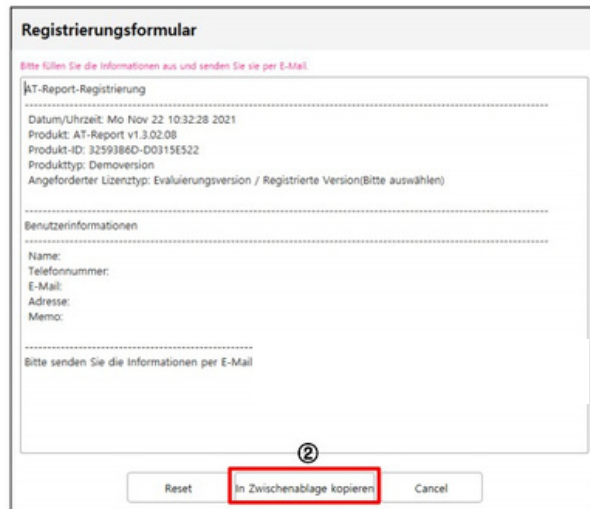


Abbildung 1.16 Zwischenablage-Symbol zum Kopieren des Anmeldeformulars und der ID

Wie man eine Lizenz für AT-Report S/W 2(DE) erhält

Confidential

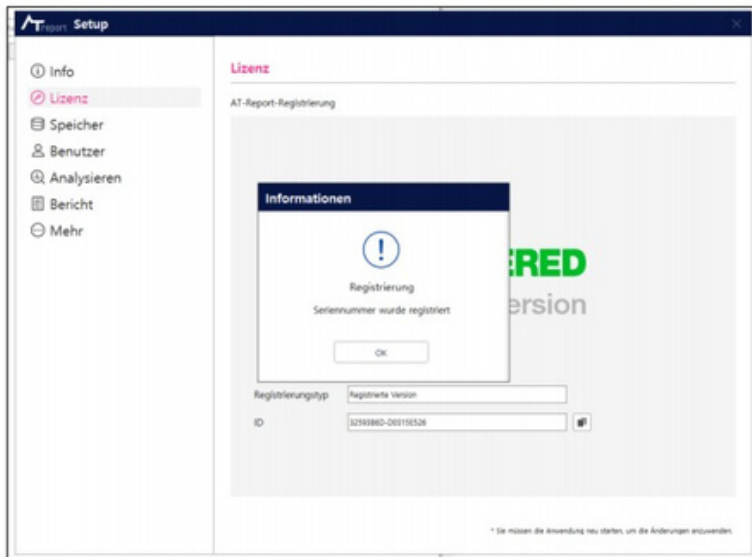

Nach Erhalt der Seriennummer, die einer von ATsens Co.,
ausgestellten ID
in das Feld
Seriennummer unter dem Feld ID ein und
klicken Sie dann auf das Symbol Bestätigen (✓), wie in
[Abbildung 1.15] gezeigt.

Wenn eine korrekte Seriennummer eingegeben wurde,
werden ein entsprechendes Symbol und eine Meldung
angezeigt, die auf dem Typ der Seriennummer basieren,
wie in [Abbildung 1.17] dargestellt.

Abbildung 1.17 Bildschirm des Eingabeergebnisses für die Seriennummer der Registrierungsversion

How to get license for AT-Report S/W 1(EN)

Confidential

The RegistrationForm window is generated when the

Figure 1.15 in the License section is clicked, as shown in ①

Once the user clicks the ② Copy to Clipboard icon in [Figure 1.16] after entering the information, the content is copied.

The copied information is to be sent to ATsens Co., Ltd. via email, and all of the entered

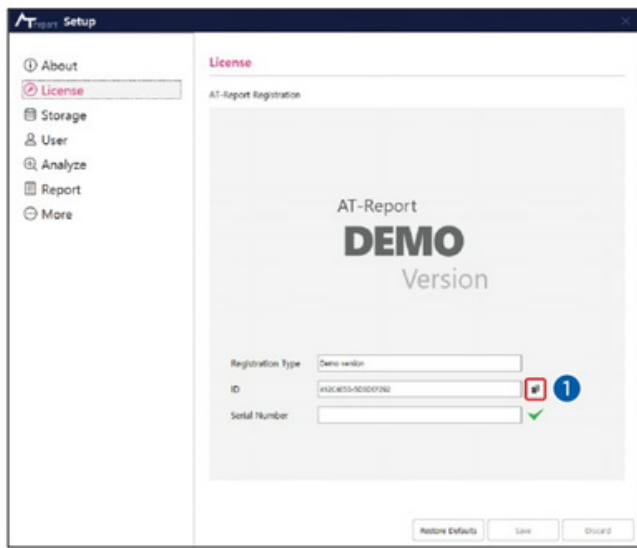


Figure 1.15 Setup License section

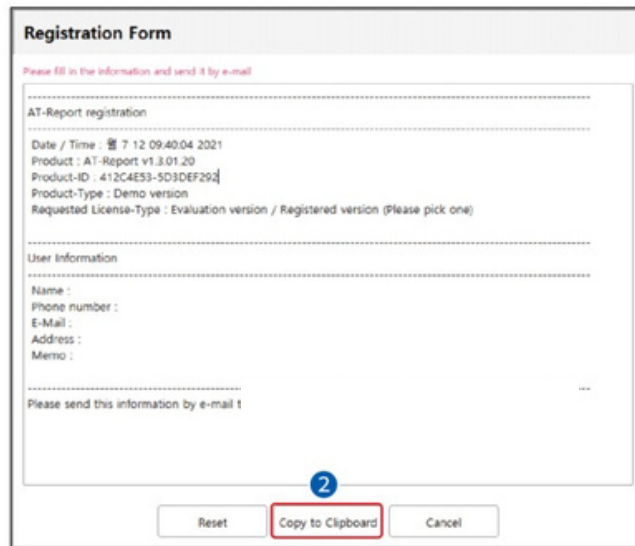


Figure 1.16 Clipboard icon for copying the Registration Form and ID

How to get license for AT-Report S/W 2(EN)

Confidential

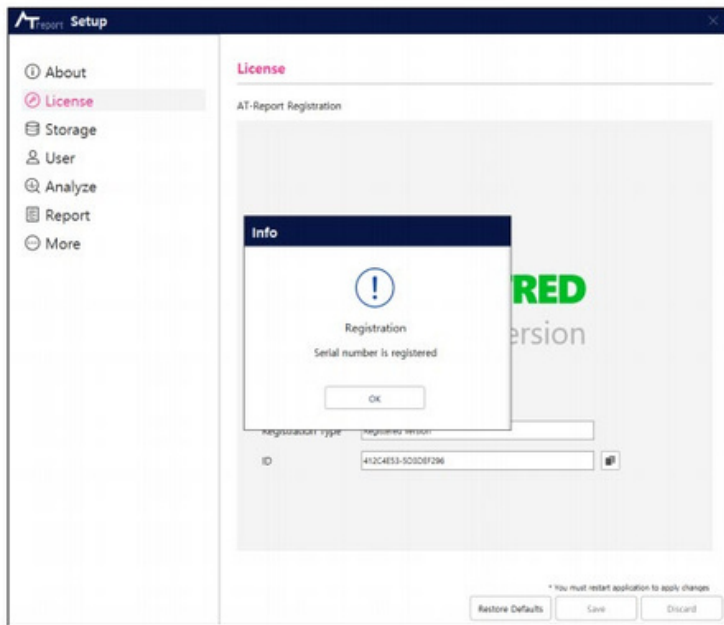


Figure 1.17 Registration version serial number input result screen

Upon receiving the serial number, which is equivalent to an ID issued by ATsens Co., Ltd.

Enter the serial number below the ID (✓) field and then click the Confirm icon (✓) as shown in [Figure 1.15]. If a correct serial number is entered, a corresponding icon and message will be displayed, based on the serial number type as shown in [Figure 1.17].

Thank you!

NEW FRONTIER
OF HEALTHCARE PLATFORM