



ALERE™ TESTPACK +PLUS HCG URINE WITH OBC
ALERE™ TESTPACK +PLUS HCG COMBO WITH OBC

RESULTS YOU CAN COUNT ON

Complete confidence for the rapid
detection of early pregnancy



TESTING WITH CONFIDENCE

Alere™ TestPack +Plus hCG offers highly sensitive test results in an easy to read '+' or '-' format. With the added benefit of unique on-board quality controls and an indicator telling you when it is time to read test results, you can have complete confidence in every test performed.

Unlike other rapid tests which only incorporate a procedural control, Alere TestPack +Plus hCG tests have built in both positive and negative controls, which ensures that the test is operating properly:

- Assures validity of results
- Monitors sample flow
- Negative control checks for interfering substances

With proven performance, Alere TestPack +Plus hCG is an ideal choice for any healthcare setting - hospital, laboratory or point-of-care:


- Only test with on board positive and negative controls for greater confidence in test results.
- Results in '+' or '-' for simple result interpretation.
- Over 99% sensitivity and specificity for trusted clinical performance.
- Available in urine or serum† test formats for flexible sampling.

ACCURATE RESULTS YOU CAN TRUST

Both Alere™ TestPack +Plus hCG Urine and Alere™ TestPack +Plus hCG Combo have shown in studies to have a relative sensitivity and relative specificity of over 99%.


		ALERE TESTPACK +PLUS HCG URINE WITH OBC	
		+	-
ANOTHER COMMERCIAL HCG COMBO TEST	+	131	0
	-	0	169

		ALERE TESTPACK +PLUS HCG COMBO WITH OBC		ALERE TESTPACK +PLUS HCG COMBO WITH OBC (SERUM)	
		+	-	+	-
ANOTHER COMMERCIAL HCG COMBO TEST	+	126	1	110	0
	-	0	168	0	76



SENSITIVITY

Alere TestPack +Plus hCG can detect hCG in samples at concentrations of 25 mIU/ml or greater.



SPECIFICITY

Alere TestPack +Plus hCG tests have been successfully evaluated to show no cross-reactivity with a variety of substances, including other hormones that can be found in urine and serum†.

EASY TESTING PROCESS


The testing process for Alere TestPack +Plus hCG is simple and can be done in three easy steps.



3


Read Test Results

Positive test results

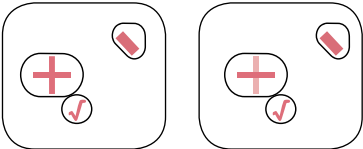

The appearance of a '+' symbol indicates a positive test result.

Negative test results

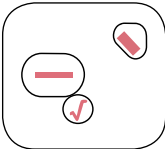

The appearance of a '-' symbol indicates a negative test result.




PRECISION YOU CAN SEE

PLUS/MINUS RESULTS
Easy to read test results.

END OF ASSAY INDICATOR
Tells you when the test is ready to read (urine only).

NEGATIVE CONTROL
Indicates the possibility of an interfering substance, causing an invalid result.

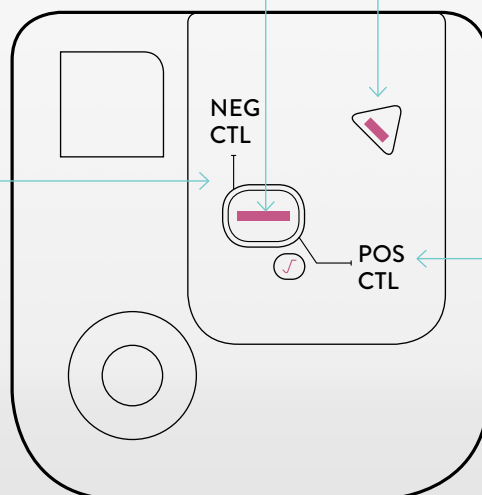

POSITIVE CONTROL
Present in all valid tests, whether hCG is present or not.

ALERE™ TESTPACK +PLUS HCG PRODUCT INFORMATION

	ALERE TESTPACK +PLUS HCG URINE WITH OBC	ALERE TESTPACK +PLUS HCG COMBO WITH OBC
Catalogue No	505798J	505805J
Kit Size	20 Tests	20 Tests
Storage	2-30°C	2-30°C
Sample Type	Urine	Urine or Serum
Time to result	At End of Assay	5 minutes
Sensitivity	25 mIU/mL hCG	25 mIU/mL hCG
Specificity	100%*	>99%*

†Serum sampling only available for Alere TestPack hCG Combo with OBC.

*Relative Sensitivity and Specificity compared to another professional pregnancy test.

