


Dengue NS1 Antigen Rapid Test Kit (Colloidal Gold Method)

The Dengue NS1 Antigen Rapid Test Kit (Colloidal Gold Method) is a rapid, qualitative immunochromatographic assay designed for qualitative detection of the presence of NS1 Antigen of dengue virus in human serum, plasma, or whole blood. The operation steps are simple and test results are interpreted visually at 15-20 minutes based on the presence or absence of visually colored lines. The diagnosis can be achieved within 1-9 days after infection, which is of great significance for prognosis and targeted treatment.


Test Procedure

- Specimen: Whole Blood/Serum/Plasma.
- Specimen collection: Specimens of all types are collected by conventional methods.
- Test method:

Add 2 drops (about 60 µl) of serum / plasma into the sample well (S) using dropper provided then add 1 drop of buffer (approximately 40 µl).

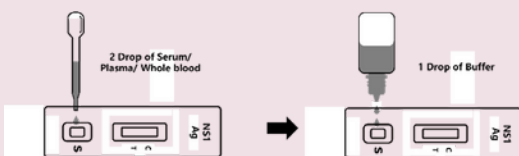

Wait for the reddish-purple line(s) to appear. The test result should be read at between 15 and 20 minutes. Result may be read up to 30 minutes.

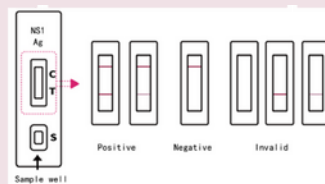

Specifications

Components	Test card/Sample diluent/ Dropper
Pack Size	25Tests/Kit; 50 Tests/Kit
Storage Conditions	2°C ~ 30°C ;Valid for 24 months
Test samples	Whole blood, Serum or Plasma
Reaction conditions	15~30°C 15~20 minutes
Special instrument	No need

Product Informations

Product Name	Cat.No	Package	Sample Diluent	Sample Type
Dengue NS1 Antigen Rapid Test Kit (Colloidal Gold Method)	BSK10S1S	25 Tests/Kit	1 × 3 mL/Tube	Whole blood, Serum or Plasma
	BSK10M1S	50 Tests/Kit	2 × 3 mL/Tube	