



# Influenza A & Influenza B Antigen Rapid Test Kit (colloidal gold method)


The Influenza A & Influenza B Antigen Rapid Test Kit (Colloidal Gold Method) is a lateral flow Immunology for qualitative detection of antigen that can specifically identify influenza A virus nucleoprotein and influenza B virus nucleoprotein to avoid detection of influenza virus nucleoprotein by chromatography. The operation steps are simple and test results are interpreted visually at 15-20 minutes based on the presence or absence of visually colored lines. The diagnosis can be achieved within 1-7 days after infection, which is of great significance for prognosis and targeted treatment.

## —Test Procedure—


Influenza A & Influenza B Antigen Rapid Test Kit (Colloidal Gold Method)




Unscrew the big lid of the sample extraction tube

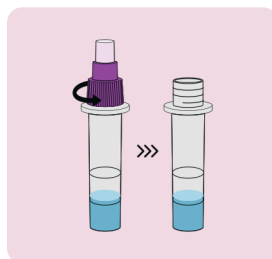

Insert the sampled swab into the solution in the sample extraction tube

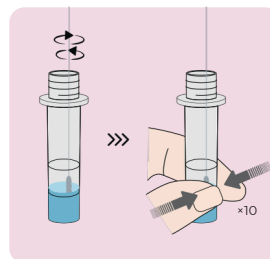


Squeeze the villi part of the swab 5 times, then remove and discard the swab

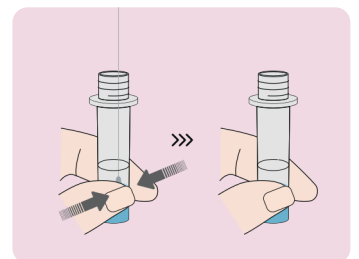


Cover the dropper lid on the sample extraction tube after sampling




Unscrew the little lid of the sample extraction tube

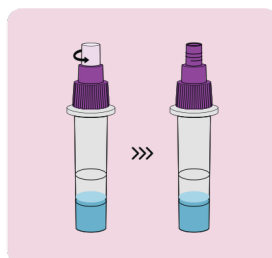

Add 5 drops (approximately 100  $\mu$ L) of the treated sample into the sample well of the test card

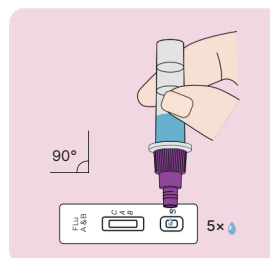

The detection results of the kit are interpreted as above

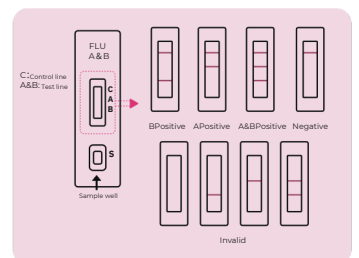



## Specifications

[www.bioer.com.cn](http://www.bioer.com.cn)

Components	Test card/Sample diluent/Sterile sampling swab
Pack Size	1 Tests/Kit; 25 Tests/Kit; 50 Tests/Kit
Storage Conditions	2°C ~ 30°C; Valid for 24 months
Test Samples	Nasopharyngeal swabs & Oropharyngeal swabs
Reaction Conditions	15~30°C 15~20 minutes
Special Instrument	No need

## Ordering Information

[www.bioer.com.cn](http://www.bioer.com.cn)

Product Name	Cat#	Package	Sample Diluent	Sample Type
Influenza A & Influenza B Antigen Rapid Test Kit (Colloidal Gold Method)	BSK09T1S	1 Test/Kit	0.5 mL/Tube	Nasopharyngeal swabs & Oropharyngeal swabs
	BSK09S1S	25 Tests/Kit	25 × 0.5 mL/Tube	
	BSK09M1S	50 Tests/Kit	50 × 0.5 mL/Tube	



**HANGZHOU BIOER  
TECHNOLOGY CO., LTD.**

Address: 1192 Bin 'an Road, Hi-tech (Binjiang) District, Hangzhou, 31005, P.R.China

Direct Phone: +86-571-87774575 (Overseas Department) Fax: +86-571-87774553

Website: <http://www.bioer.com.cn> Email: [overseas@bioer.com.cn](mailto:overseas@bioer.com.cn) E-Date: 2022.10

\* All rights reserved BIO K09-2210