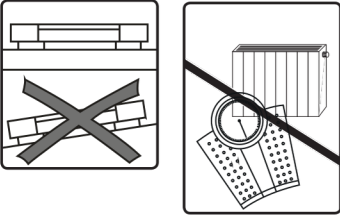
DE	Mechanische Personenwaage Gebrauchsanleitung	DA	Mekanisk personvægt Brugsanvisning
EN	Mechanical personal bathroom scale Instruction for use	SV	Mekanisk personvåg Bruksanvisning
FR	Pèse-personne mécanique Mode d'emploi	NO	Mekanisk personvekt Bruksanvisningen
ES	Báscula mecánica Instrucciones para el uso	FI	Mekaaninen henkilövaaka Käyttöohje
IT	Bilancia pesapersona meccanica Istruzioni per l'uso	CS	Mechanická osobní váha Návod k použití
TR	Mekanik terazi	SK	Mechanická osobná váha
RU	Kullanma Talimatı Механические напольные весы	SL	Návod na používanie Mehanična osebna tehtnica
PL	Instrukcja po zastosowaniu Mechaniczna waga łazienkowa	HU	Navodila za uporabo Mechanikus személymérés
NL	Instrukcja obsługi Mechanische personenweegschaal	RO	Használati útmutató Cântar mecanic pentru persoane
PT	Gebruikshandleiding Balança de casa-de-banho mecânica	AR	میزان الأشخاص الميكانيكي إرشادات الاستخدام
EL	Instruções de utilização Μηχανική ζυγαριά σωματικού βάρους		
Οδηγίες χρήσεως			

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany • www.beurer.com
www.beurer-gesundheitsratgeber.com • www.beurer-healthguide.com

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	Max.	d
MS 50	135 kg	1 kg

EN	Getting started	DA	Ibrugtagning
FR	Mise en service	SV	Börja använda vågen
ES	Puesta en marcha	NO	Bruk
IT	Messa in funzione	FI	Käyttöönotto
TR	İlk çalıştırma	CS	Uvedení do provozu
R	Ввод в эксплуатацию	SK	Uvedenie do prevádzky
PL	Uruchomienie	SL	Vkllop
N	Ingebruikname	HU	Üzembe helyezés
L	Colocação em funcionamento	RO	Punerea în funcțiune
P	Έναρξη λειτουργίας	AR	التشغيل
T			
EL			

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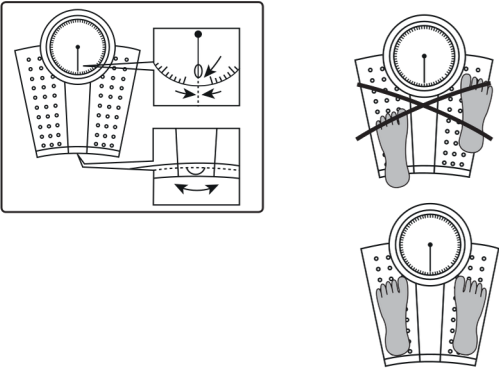
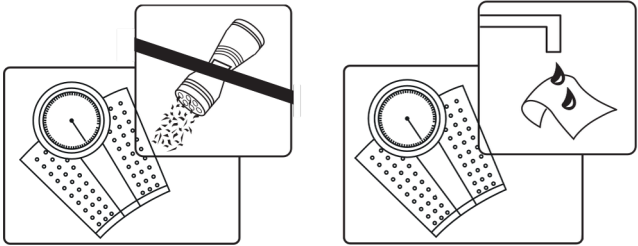
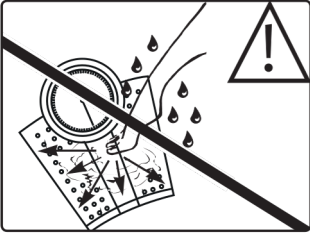
EN Important instructions

- Place the scales on a firm level surface; a firm surface is essential for correct measurement
- Protect your personal scales from impact with hard objects, moisture, dust, chemicals, toiletries, liquid cosmetics, g temperature fluctuation and closeness to sources of heat (open fires, radiators).
- Repairs may only be performed by Korona Customer Service or by accredited retailers.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scales in water or rinse it under running water. Do not use aggressive or abrasive cleaning products.
- Storage: Do not place any objects on the scale when not in use.
- The scale is not intended to be used for commercial purposes.
- Remove transport safety catch.
- Never step onto the outer edge of the scales with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Dispose of packaging in an environmentally friendly manner.

Warranty/service
In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list “service international”.
In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:
1. The warranty period for BEURER products is either 5 years or- if longer- the country specific warranty period from date of Purchase.

- 2. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
- 3. Repairs (complete unit or parts of the unit) do not extend the warranty period.
- 4. The warranty shall not be valid for damages because of
 - a. improper treatment, e.g. nonobservance of the user instructions.
 - b. repairs or tampering by the customer or unauthorised third parties.
 - c. transport from the manufacturer to the consumer or during transport to the service centre.
 - d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
- 5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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