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Rx ONLY

MADE IN
THE U.S.A.

©2019 Bionix Development Corp. | 5154 Enterprise Blvd., Toledo, Ohio 43612

Phone: 800.551.7096 | Fax: 800.455.5678 | Email: bionix@bionix.com

RM95-0006 | Rev. C | Patent#s D524,939 & D546,948

bionix.com

THE LIGHTED EAR CURETTE™ WITH MAGNIFICATION - USER INSTRUCTIONS

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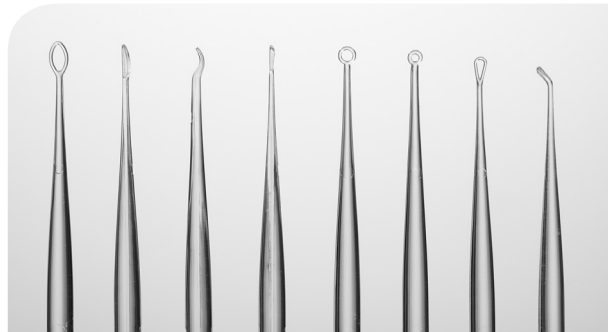
#2245 - VARIETY PACK (INCLUDES 7 TIP STYLES)

bionix®

THE LIGHTED EAR CURETTE™ WITH MAGNIFICATION - USER INSTRUCTIONS

Assembly/Disassembly:

1. To assemble the lighted ear curette, insert the end of the curette into the light source by aligning the pegs on the curette end with the grooves inside the light source. Push down and twist FIRMLY clockwise until rotation stops. This will lock the curette into place and activate the light.
2. To attach the magnification lens, snap the lens onto the neck of the light source so that the top of the lens angles towards the ear curette tip.
3. To disassemble, push down on the curette and rotate counter clockwise to remove the curette from the light source. The light will go off when the curette is removed.

Instructions For Cerumen Removal

The intended use of the Lighted Ear Curette is to remove cerumen from the ear canal.

1. Use an otoscope to examine the ear canal and determine cerumen location.
2. Select the appropriate style lighted ear curette for the type of patient and cerumen occlusion. Assemble according to instructions.
3. Use lighted ear curette to remove cerumen from the ear canal.
4. Repeat otoscope exam. Determine location of any remaining cerumen.
5. Repeat curettage procedure until sufficient cerumen is removed to allow adequate visualization of the tympanic membrane.
6. Disassemble the lighted ear curette, and discard the used curette in the usual fashion. Save the light source for the next curettage procedure. LEC Tips are intended for single patient use only.

CAUTION: Discontinue curettage immediately if bleeding, irritation, or other trauma to the ear canal or tympanic membrane occurs.

CAUTION: This product is a spring-loaded device; improper assembly may cause ejection of the ear curette.

CAUTION: DO NOT hold device by light source as accidental ejection of curette may occur if not assembled properly.

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