



Dosiertabelle / Dosage table

fertige Gebrauchslösung ready-to-use solution	0,25 %	0,5%	1,0%	1,5%	2,0%	3,0%	4,0%
1 L	2,5 ml	5 ml	10 ml	15 ml	20ml	30ml	40 ml
2L	5ml	10 ml	20ml	30 ml	40 ml	60ml	80ml
3L	7,5m l	15 ml	30ml	45 ml	60ml	90 ml	120 ml
4L	10 ml	20ml	40 ml	60 ml	80ml	120 ml	160 ml
5L	12,5 ml	25 ml	50ml	75 ml	100m l	150m l	200ml
SL	20ml	40 ml	80 ml	120 ml	160m l	240 ml	320ml
10L	25m l	50 ml	100ml	150m l	200 ml	300ml	400ml
16 L	40 ml	80 ml	160 ml	240 ml	320ml	480ml	640m l
20L	50ml	100 ml	200 ml	300 ml	400ml	600ml	800 ml
30L	75m l	150m l	300ml	450 ml	600ml	900 ml	1200 ml

Herstellung gebrauchsfertiger Desinfektionslösungen:
Die Anwendungslösung ist aus Konzentrat und Leitungswasser herzustellen. Um 5 Liter Anwendungslösung mit einer 2 %igen Konzentration herzustellen, müssen 100 ml des Konzentrats in 4,9 Liter Leitungswasser hinzugefügt werden. Die Richtlinien und Einwirkzeiten des Herstellers sind zu beachten.

Preparation of a ready-to-use solution of disinfectants:
The ready-to-use solution consists of concentrate and tap water. In order to prepare 5 liter ready-to-use solution with a concentration of 2 % you have to add 100 ml of concentrate into 4,9 liters of tap water. The guidelines and exposure times of the manufacturer should be followed.