

Tri-Wave MD Combined Treatment Protocol Chart

COMBINED TREATMENT	PROGRAMME	PRE-SET PROTOCOL			TIME	JOULES
		BLUE 415NM	RED 633NM	NIR 830NM		
FACIAL TREATMENT: SKIN REJUVENATION	POST TREATMENT REJUVENATION				10 MINUTES	96
FACIAL TREATMENT: ANTI-BACTERIAL	POST TREATMENT ANTI-BACTERIAL				10 MINUTES	57
FACIAL TREATMENT: CALMING	POST TREATMENT HEALING				10 MINUTES	33
RESURFACER / SKIN PEEL: SKIN REJUVENATION	POST TREATMENT REJUVENATION				10 MINUTES	96
RESURFACER / SKIN PEEL: ANTI-BACTERIAL	POST TREATMENT ANTI-BACTERIAL				10 MINUTES	57
RESURFACER / SKIN PEEL: CALMING	POST TREATMENT HEALING				10 MINUTES	33
MICRO / HYDRA DERMBRASION: SKIN REJUVENATION	POST TREATMENT REJUVENATION				10 MINUTES	96
MICRO / HYDRA DERMBRASION: ANTI-BACTERIAL	POST TREATMENT ANTI-BACTERIAL				10 MINUTES	57
DERMAPLANNING	POST TREATMENT REJUVENATION				10 MINUTES	96
SKIN NEEDLING	POST TREATMENT ANTI-BACTERIAL				10 MINUTES	57
MESOTHERAPY	POST TREATMENT REJUVENATION				10 MINUTES	96
MICROPIGMENTATION	POST TREATMENT ANTI-BACTERIAL				10 MINUTES	57
ELECTROLYSIS	POST TREATMENT HEALING				10 MINUTES	33
CRYOTHERAPY	POST TREATMENT REJUVENATION				10 MINUTES	96
PLASMA TREATMENTS	POST TREATMENT HEALING				10 MINUTES	33
SKIN TIGHTENING: FACE / NECK / BODY	POST TREATMENT REJUVENATION				10 MINUTES	96
LASER AND IPL	POST TREATMENT HEALING				10 MINUTES	33
FRACTIONAL TREATMENTS	POST TREATMENT HEALING				10 MINUTES	33
PRP THERAPY	POST TREATMENT HEALING				10 MINUTES	33
WRINKLE RELAXING TREATMENT	POST TREATMENT REJUVENATION				10 MINUTES	96
DERMAL FILLERS	POST TREATMENT HEALING				10 MINUTES	33
ABLATIVE AND SURGICAL PROCEDURES	POST TREATMENT HEALING				10 MINUTES	33

With combination protocols, apply Dermalux immediately post treatment. For certain sensitive and chronic indications or in combination with more aggressive procedures, pre Dermalux application (1 - 2 weeks prior) is recommended to prepare the skin. Additional post treatment application is also recommended.

*Following any procedure, if the skin presents with excessive erythema, as irritated, aggressed or there is incidence of adverse response, choose the Post Treatment Healing over the recommended programme.