AND

Digital Thermometer

Model UT-103

Instruction Manual

Original

English

Mode d'Emploi

Français

Manual de Instrucciones

Español

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Itaiano

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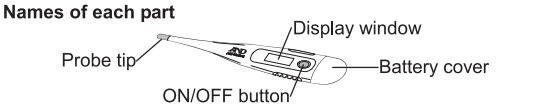
1WMPD4002219

Preliminary remarks
☐ This device conforms to the European Directive 93/42 EEC for Medical Products. This is proved by the CE conformity marking **CE**⁰⁴³⁴ (0434 : The identification number of the Notified Body)
☐ Environment for use: The device is designed for use indoors.

Precautions
☐ Precision components are used in this device. Extreme temperature, humidity, direct sunlight, shock or dust should be avoided.
☐ Measurements may be affected if the device is used near a television, microwave oven, cellular telephone, X-ray machine or other devices that emit electromagnetic waves.
☐ The thermometer should only be used under the supervision of an adult.
☐ Used equipment, parts and batteries are not treated as ordinary household waste, and must be disposed of according to the guidelines in your area.
☐ Before and after each use, clean the thermometer gently using a soft cloth moistened with water-diluted isopropyl alcohol or cold soapy water.
☐ Do not immerse the thermometer in water.
☐ Do not sterilize the thermometer by boiling or gas or steam autoclaves.

Symbols	Function / Meaning	Symbols	Function / Meaning
	Standby and power on.		Direct current
SN	Serial number		Date of manufacture
	Type B: The device is designed to provide special protection against electrical shocks.		
	Last reading stored in memory		
	The battery is low. Replace the battery with a new one.		
H	The reading is over 43 °C.		Do not use the thermometer to measure the temperature of the object.
Lo	The reading is below 32 °C.		
CE ⁰⁴³⁴	EC directive medical device label		WEEE label
	Manufacturer		EU-representative
	Consult instructions for use		

Names of each part

How to take your temperature
Note: Do not walk, run or talk during temperature measurement.
1. Press the ON/OFF button to turn the thermometer on. The thermometer beeps and displays **188.8°C** (or **188.8°F**) for about 2 seconds. The last temperature taken is displayed with an "M" mark in the right corner for about 1 second.

2. On the display, 37.0°C (or 98.6°F) appears.

Switching between Fahrenheit (°F) and Celsius (°C):
(1) Press the ON/OFF button to turn the thermometer off.
(2) Press and hold the ON/OFF button for 5 seconds. The display changes to the other measurement scale.

3. When "Lo" and flashing °C (°F) are displayed, the thermometer is ready for temperature measurement. ("Hi" or room temperature is displayed when the room temperature reaches a certain temperature as shown below.)

Room temperature	<32.0°C (89.6°F)	32.0°C to 42.9°C	>42.9°C (109.2°F)
Display	Lo	Room temperature	Hi


Note: The thermometer can not measure a person's temperature when the room temperature is higher than the body temperature.

4. The thermometer beeps when the measurement is completed.

5. Press the ON/OFF button to turn the thermometer off.
(The thermometer automatically turns off, about 8 minutes ± 2 minutes after use.)

English 1

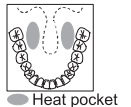Taking an oral temperature
Place the probe tip under the tongue as near as possible to a heat pocket.
Approximate measuring time: 1 minute.

Taking an axillary temperature (under the arm)
1. Wipe the underarm with a dry towel.
2. Place the probe tip under the arm so the tip touches the skin with the thermometer perpendicular to the body.
3. Position the arm across the chest so the probe tip is well covered by the arm.
Approximate measuring time: 2-3 minutes.

Note: Axillary temperatures are generally 0.5°C /1.0°F lower than oral readings.

Taking a rectal temperature
1. Apply a water-soluble lubricant to the probe tip.
2. Gently insert the probe (maximum 1/2") into the rectum.
Approximate measuring time: 1 minute.

Note: Rectal temperatures are generally 0.5°C /1.0°F higher than oral readings.

Maintenance
☐ Store the thermometer in the protective case when not in use.
☐ Do not store the thermometer where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
☐ Do not drop or subject the thermometer to strong shocks.
☐ Do not attempt to disassemble the thermometer, except to replace the battery.

Replacing the battery
Replace the battery with a new one when appears in the lower right corner of the display. Battery type: 1.55V, LR41 alkaline or SR41 silver oxide type
1. Remove the battery cover.
2. Use a non-metal pointed tool to remove the battery and insert the new battery with the positive (+) side facing up.
3. Slide the battery cover back into place.

Note: Keep the battery out of the reach of children.
Be careful not to swallow the battery cover accidentally.

Specifications	
Measurement range	32.0°C (89.6°F) to 42.9°C (109.2°F)
Low temperature display	Temperature<32.0°C (89.6°F): display: Lo°C (°F)
High temperature display	Temperature>42.9°C (109.2°F): display: Hi°C (°F)
Temperature display	0.1°C per (0.1°F per)
Accuracy	35.5°C (95.9°F) to 42.0°C (107.6°F): ±0.1°C (0.2°F) Other ranges: ±0.2°C (0.3°F)
Fever beep Sound	[>37.5°C (99.5°F)] 30 short beep sounds
Normal beep sound	[≤37.5°C (99.5°F)] 10 long beep sounds
Auto power off	About 8 minutes ± 2 minutes
Battery	LR41 or SR41
Battery life	2 years
Dimensions	130.36 x 18.67 x 10.55 mm
Operating conditions	Temperature: 10°C to 40°C (50.0°F to 104.0°F) Humidity: 30%RH to 85%RH
Storage conditions	Temperature: -10°C to 60°C (14.0°F to 140.0°F) Humidity: 25%RH to 90%RH

EMC table information is listed on our website:
http://www.aandd.jp/products/manual/medical/emc_en.pdf

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