Cardiology Stethoscopes

Littmann®
Cardiology IV™
Stethoscope

3M

Table of Contents

ⒸⒹ English	pp. 1	ⒸⒿ Cesky	pp. 31
ⒻⒼ Français	pp. 3	ⒸⓀ Slovensky	pp. 33
ⒹⒺ Deutsch	pp. 5	ⒸⓁ Slovenscina	pp. 35
ⒾⓉ Italiano	pp. 7	ⒺⒺ EESTI	pp. 37
ⒺⓈ Español	pp. 9	ⒻⓉ Lietuviskai	pp. 39
ⒼⒻ Nederlands	pp. 11	ⒻⓇ Română	pp. 41
ⒺⒺ Svenska	pp. 13	ⒻⓇⓁ Русский	pp. 43
ⒹⓀ Dansk	pp. 15	ⒻⓇⓁ Українська	pp. 45
ⒼⓇ Norsk	pp. 17	ⒻⓇⓇ HRVATSKI	pp. 47
ⒻⓁ Suomenkielinen	pp. 19	ⒻⓇⓈ Srpski	pp. 49
ⒻⓉ Português	pp. 21	ⒻⓇⓇ Turcke	pp. 51
ⒻⓉ Português	pp. 23	ⒸⒼ 中文	pp. 53
ⒼⓇ Ελληνικά	pp. 25	ⒸⒼ 中文	pp. 55
ⒻⒻ Polski	pp. 27	ⒸⓇ ئىبرع ەغل	pp. 57
ⒻⓇ Magyar	pp. 29		

Features and Benefits

- Outstanding acoustic performance
- Single-piece tunable diaphragm allows better audibility of high frequency sounds (adult side, firm pressure)
- Easy to clean, single-piece tunable diaphragms
- Versatile adult and pediatric two-sided chestpiece
- Next generation longer-life tubing
- Dual-lumen tubing provides two sound paths in one tube
- Pediatric side converts to open bell
- 7 year warranty
- Included in box:
 - Snap Tight Soft-Sealing Eartips - Large
 - Snap Tight Soft-Sealing Eartips - Small
 - Snap Tight Firm Eartips
 - Nonchill Bell Sleeve

Explanation of Symbols


Consult instructions for use.


Not made with natural rubber latex.

Instructions for Use


Insert eartips in a forward direction.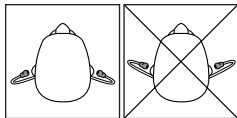

Adjust the Headset Tension

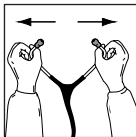To reduce the spring tension, pull the eartubes apart.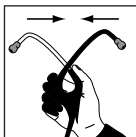

To increase the spring tension, squeeze the eartubes together.


Select the Active Side of the Chestpiece

Only one side of the chestpiece is acoustically active at a time, identified by a mark on the stem.


Rotate the chestpiece to change which side is active.

Listen to Low and High Frequency Sounds with Littmann Tunable Diaphragms on Each Side of the Chestpiece

Use the large side for adults and the small side for pediatric or thin patients, around bandages and carotid assessment.

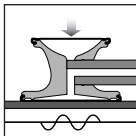
Low frequencies: apply light pressure to the chestpiece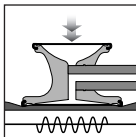

High frequencies: apply firm pressure to the chestpiece

Convert to a Traditional Open Bell

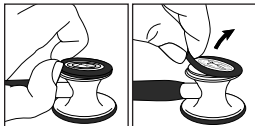Pinch the rim of the diaphragm on the small side; pull and lift from the chestpiece.

Attach the sleeve by placing one side on the edge and stretching it over the chestpiece.


To install diaphragm, remove the nonchill sleeve. Insert the edge of the diaphragm into the groove and roll the rim around the chestpiece.

Cleaning, Disinfection and Storage

Clean your stethoscope between exams using a 70% isopropyl alcohol wipe or a disposable wipe with soap and water. A 2% bleach solution may be used to disinfect your stethoscope; however, the tubing may experience a slight discoloration after exposure to bleach. The tunable diaphragms, nonchill bell sleeve and eartips may be removed for thorough cleaning.

To remove the eartips from the headset, pull firmly. To assemble the eartips, push the small side of the eartip firmly onto the eartube until it snaps fully into place. Ensure all parts and surfaces are dry before reassembly. Be sure the eartips are firmly attached to the headset before inserting them into your ears.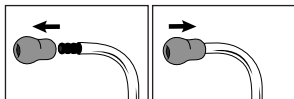

Do not immerse your stethoscope in any liquid.
Do not subject your stethoscope to any sterilization process.
Avoid storing your stethoscope in extreme heat.
To prevent staining of stethoscope tubing, avoid contact with pens, markers, newspaper, or other printed materials.

Littmann Stethoscope Service and Warranty Program

Please register your stethoscope online at: <http://www.littmann.com/warranty-registration>.

The Littmann Cardiology IV stethoscope is warranted against any defects in material and workmanship for a period of seven (7) years. Within the warranty period, repairs will be made without charge upon the return of the stethoscope to 3M, except in cases of obvious abuse or accidental damage.

For service and repair in the U.S.A., please visit www.littmann.com/service or call 1-800-292-6298. If you are outside of the U.S.A., please visit www.littmann.com for the contact information of your local 3M office.

For more information, visit www.littmann.com.

3M, Littmann, the L Littmann logo, the L, and Cardiology IV are trademarks of 3M.

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© 2015, 3M. All rights reserved.
Made in U.S.A. by  **3M Health Care**,
(U.S.A.) 1-800-228-3957 Visit our website www.littmann.com
3M Deutschland GmbH