105X145MM

ORTHO

MECHANIK

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WARNING

When using Creams, Lotions or Oils please avoid direct contact with the massage gun.

The paint on the Orthogun may react with these substances and get damaged.

Only expose the metal-colored massage-heads to these substances.

First steps

- 1. Please charge the device for 5 hours before the first treatment.**
- 2. Please do not turn on while it is charging.**
- 3. Choose the massage-head and insert it fully.**
- 4. Press the button until the OrthoGun starts running.**
- 5. A quick press of the button will change the gear.**
- 6. A long press of the button will turn the device off again.**
- 7. Please be aware of our “use and operations” warnings (last page).**

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Thank you for purchasing the Orthogun.

We appreciate the trust you have placed in our brand and will do our best to deliver a great experience.

Your Orthogun can help you reduce post-workout muscle pain, warm-up your muscles before exercise or just help you relax at the end of a stressful day.

For more information on how to get the most out of your Orthogun, please visit

www.orthomechanik.de.

If you have any questions, contact us at

info@orthomechanik.de.

We wish you a lot of success with your OrthoGun.

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

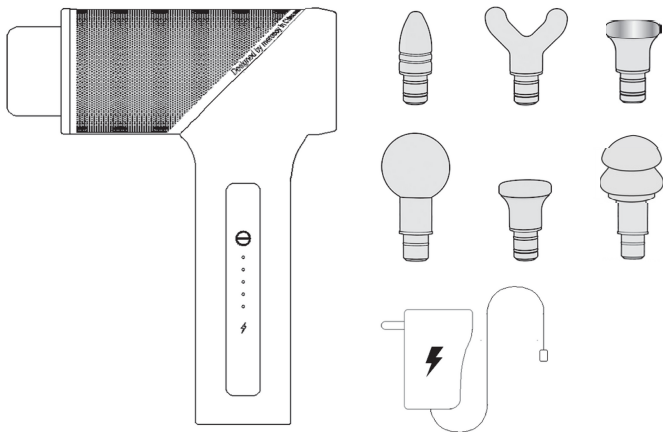
1 PACKING LIST

Technical Specifications:

Model	OrthoGun 2.0, model miniS
Battery Capacity	3400mAh/ 42Wh
Power	15-160 w
Frequency	1200 rpm 1600 rpm 2200 rpm 2800 rpm 3200 rpm
Torque	0~7.5kg/cm
Intelligent Mode	10 Mins Smart Timing

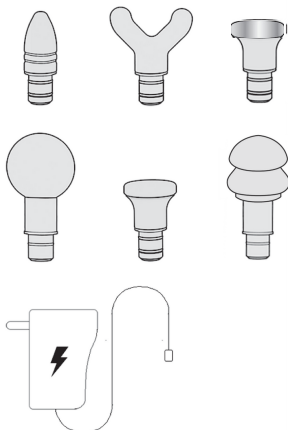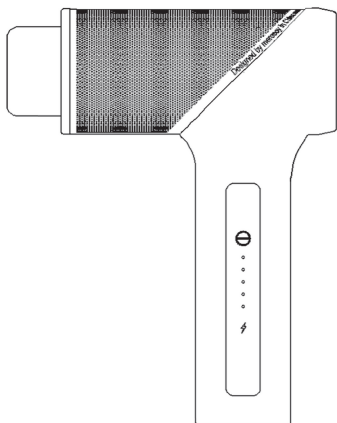1 INHALT

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
Modell	OrthoGun 2.0, model mini S
Akkuleistung	3400mAh/ 42Wh
Leistung	15-160 w
Frequenz	1200 U/min 1600 U/min 2200 U/min 2800 U/min 3200 U/min
Drehmoment	0~7.5kg/cm
Timer	10 Mins Smart Timing

2

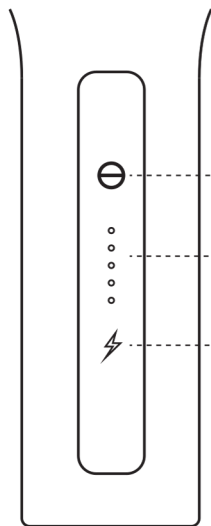USER INSTRUCTIONS

**1 Button**

- Long Press to Start and Stop
- Short Press to switch gears

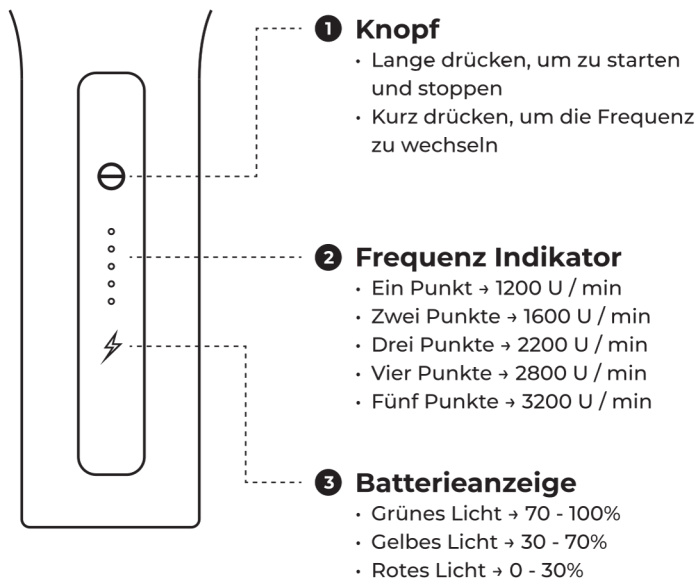
2 Gear Indicator

- One dot → 1200 rpm
- Two dots → 1600 rpm
- Three dots → 2200 rpm
- Four dots → 2800 rpm
- Five dots → 3200 rpm


3 Battery Indicator

- Green Light → 70 - 100%
- Yellow Light → 30 - 70%
- Red Light → 0 - 30%

2 ANLEITUNG


3 HEADS

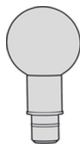BALL HEAD

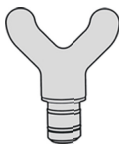For large muscle groups
(quads, glutes, etc.)


FORK HEAD

For mid-sized muscle groups
(neck spinal, calves, etc.)

CONE HEAD

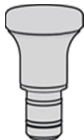For pinpoint muscle
treatment, trigger points, and
small muscle groups (feet,
wrist, etc.)


FLAT HEAD

For general use on large
muscle groups.

METAL HEAD

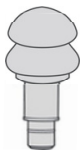For use with massage oils.
(Keep oil off painted areas)


SOFT HEAD

For use on sensitive areas
like the neck.

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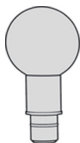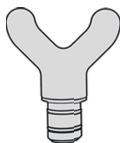
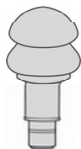
DEVICE WARNINGS

- Keep away from liquids or heat sources.
- Keep ventilation ports free from dust and debris.
- Do not remove screws or attempt to disassemble.
- Do not operate continuously for more than one hour. Allow the device to cool down for 30 minutes before reusing.
- Only use the original charger.
- Unplug the charger from the device after charging or prior to use. Do not use it while charging.
- For adult use only.
- If you experience pain or discomfort, then immediately stop using the device.
- For adult use only. Do not use if you are injured. Consult your physician prior to use.

USE AND OPERATION WARNINGS

- If you experience pain or discomfort, then immediately stop using the device.
- Consult your physician prior to use if the following applies: Osteoporosis patients, anyone with heart problems wearing an electronic medical device, pregnant women, anyone with physical trauma or any other medical complications.
- Only use on a dry clean surface of the body over clothing by lightly pressing and moving across the body for approximately 60 seconds per region.
- Use the Orthogun on the body's soft tissue only. Do not use on head or bones.
- Bruising can occur even with low-frequency and pressure. Check treated areas frequently and immediately stop at first sign of pain or discomfort.
- Keep fingers, hair or any other body part away from the shaft of the device.

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Please note that the battery is permanently installed; in case you want to change the battery, please get in touch with our service team at **info@orthomechanik.de**

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Orthomechanik GmbH