

Oxivir Plus Spray

Oxivir Plus Spray: Broad Spectrum Cleaner & Disinfectant

Description

Diversey **Oxivir Plus Spray** is a ready to-use liquid cleaner disinfectant for the cleaning and disinfection of all water-resistant hard surfaces and some soft furnishings.

Key properties

Hydrogen Peroxide based acidic formulation

Effective against a wide spectrum of micro-organisms including bacteria, viruses, fungi and yeasts

Effective cleaner

Fragrance-free

Based on patented technology; accelerated hydrogen peroxide

Benefits

Cleans and disinfects in one step, reduction of working time

Extensively tested against European norms and proven broad spectrum disinfection:

-Bactericidal, Virucidal, Fungicidal and Yeasticidal

Excellent cleaning performance offering a streak free finish, descaling and lime limitation

Suitable for use on water-resistant hard surfaces and some soft furnishings

Good user safety profile

Good environmentally profile. Active breaks down into oxygen and water

Suitable for use in all kind of application areas, eg in high risk areas in Healthcare

Use instructions

Dosing:

Use undiluted

Application:

Spray cleaning and disinfection:

- 1.Remove loose dirt
- 2.Spray onto surfaces
- 3.Clean with clean cloth, pre-wetted with product
- 4.Spray again for at least 5 minutes and wipe to dry

Important:

Do not mix with other products. Do not use on water- or acid sensitive surfaces.

Technical data

Appearance:

Relative density [20°C]:

pH-value in use:

Clear colourless liquid

1.000

1.8

The above data is typical of normal production and should not be taken as a specification



Oxivir Plus Spray

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Material Safety Data Sheet. Store in original container. Avoid extremes of temperature. Reacts with oxydisable substances.

Product compatibility

Under recommended conditions of use Diversey **Oxivir Plus Spray** is suitable for use on most materials commonly encountered.

Environmental information

The surfactants used in this product are ultimately biodegradable in compliance with the requirement of the EU Detergent Regulation, EC 648/2004.

Microbiological data

Efficacy proven by compliance with: EN1276, EN13697, EN1650, EN14476, EN13624 and EN13727.

Available pack sizes

Article code 7519553 7519554 7519559 **Oxivir Plus Spray** 6x0.75L Spray bottle with foam trigger

Pack size	Pack type
-----------	-----------

Oxivir Plus Spray 6x0.75L	Spray bottle with foam trigger
----------------------------------	--------------------------------

Oxivir Plus Spray 6x0.75L	Spray bottle with foam trigger
----------------------------------	--------------------------------

Diversey Europe Operations B.V.

De Boelelaan 32

1083 HJ Amsterdam

The Netherlands

Tel +31 (0)20 258 9700

www.diversey.com

P85709XFEN-01