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User manual

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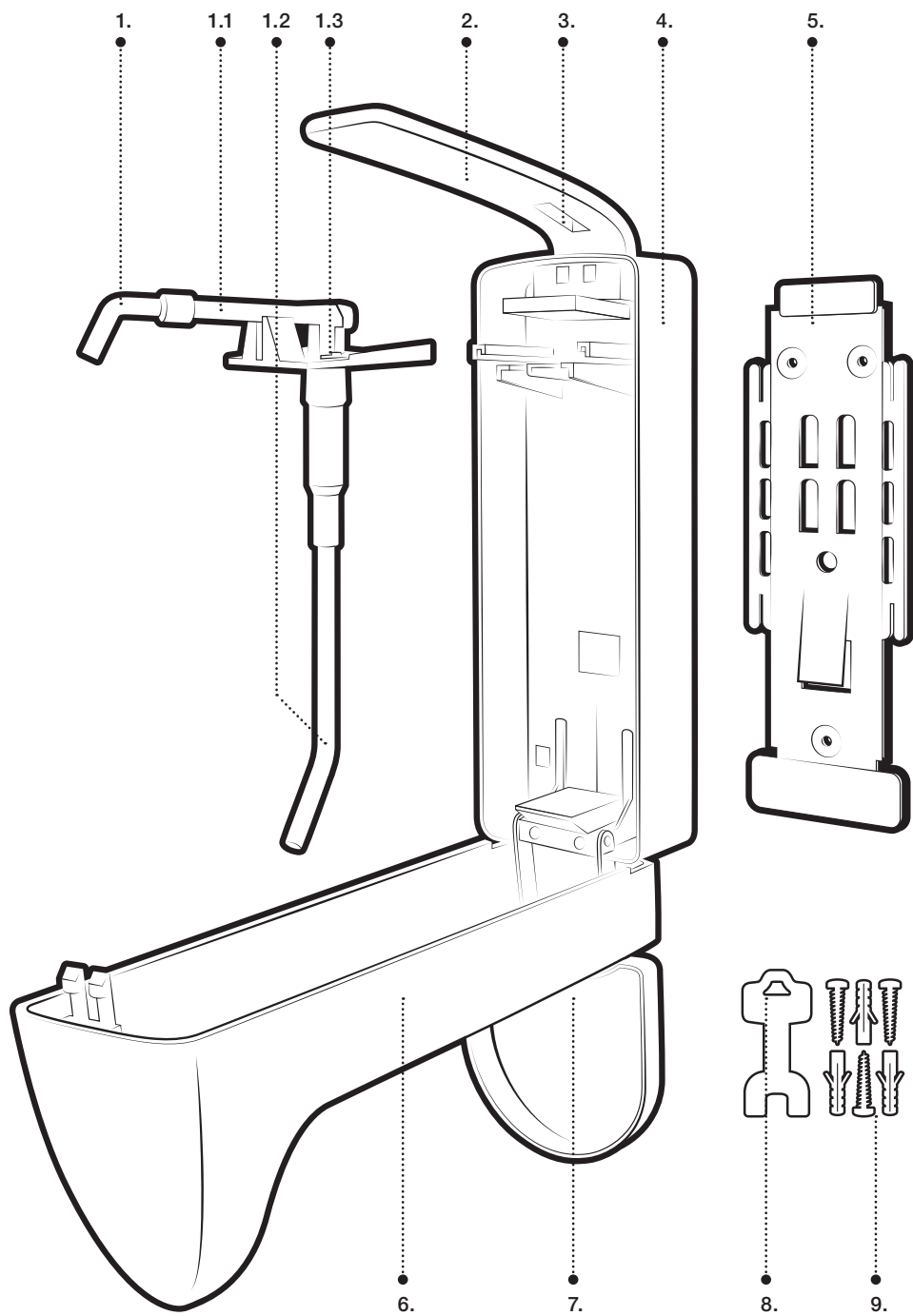
GB

Congratulations!

You bought an approved dispenser which is easy to handle and assures you working processes as used.

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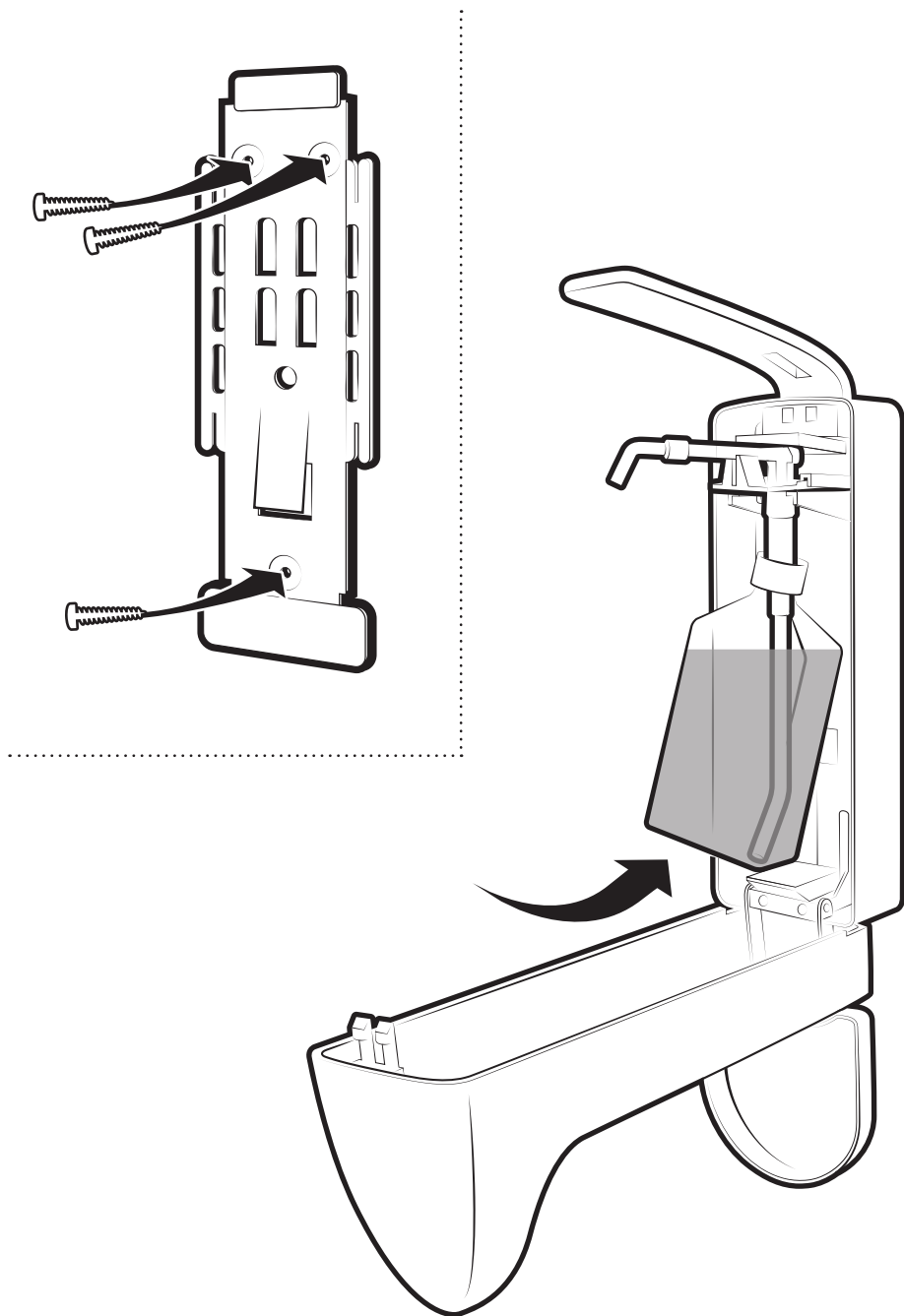
- 1. Pump
- 1.1 outlet pipe
- 1.2 intake pipe
- 1.3 dosing fork
- 2. arm lever
- 3. key opening
- 4. plastic housing
- 5. wall panel
- 6. Front cover
- 7. Drip dray
- 8. Key
- 9. mounting kit

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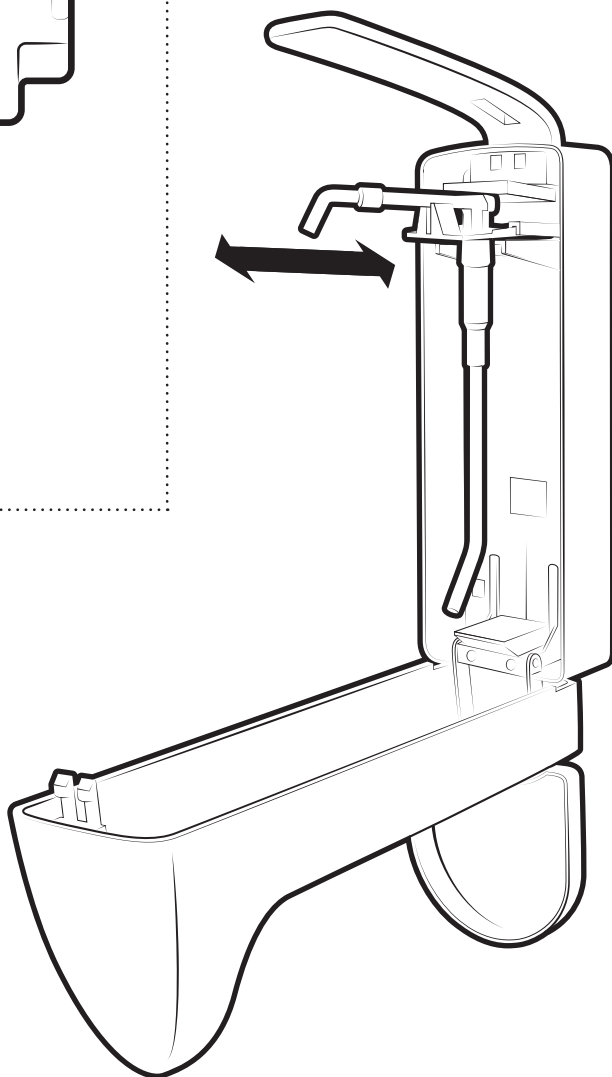
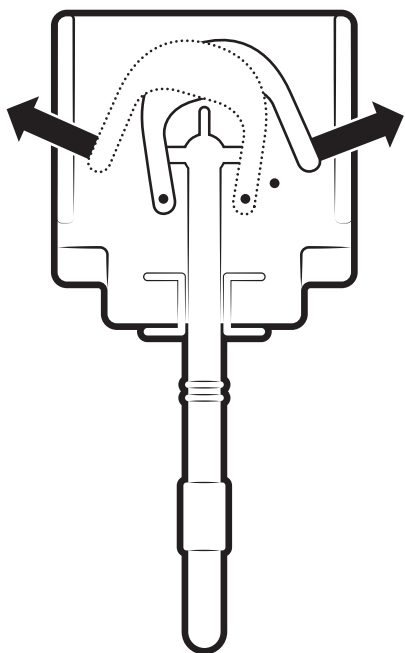
GB

1. Mounting of Dispenser

Please identify the correct position of the dispenser before it is mounted.

The following minimum clearances must be observed:

- Above the dispenser:
keep a space of minimum 40 mm
- Below the dispenser:
min. 180mm
- Take the dispenser out of package.
- Open the dispenser with the included key.
- Press locking on the wall plate and pull wall plate downward out of the body.
- Insert dowels in predrilled holes and fix the wall holder with screws.
- Push dispenser body top down over wall holder until you hear the locking snap in.
- Push full bottle bottom-up over the dispenser pump and insert in the body.
- Place the drip tray at the front of the dispenser.
- Attention! The drip tray sits very tight on the dispenser to prevent possible vandalism and theft.

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2. Important notice

Dosing of liquid:

Delivery is with a max. volume of 1,8 ml per pass. By changing the dosing fork you can reduce the volume.

- By pushing right:
approx. 1,2 ml per pass.
- By pushing left:
approx. 0,8 ml per pass.

Maintenance:

To ensure proper function of the dispenser and to ensure the basis of recommendations for infection prevention and hygiene, the dispenser and the pump should be regularly cleaned and if necessary disinfected:

- For cleaning the pump first remove bottle, then press locking at the wall plate and pull dispenser upwards.
- Hold lever in upper position and pull out the pump backwards. For inserting again go reverse.
- Use gentle cleaning agents only.
- If the pump is blocked (e.g. no use for a longer time), remove bottle and rinse thoroughly with hot water until the work is perfect again.

Note: Plastic pumps are spare and wear parts and should be replaced at an interval of six months.

Attention: Neither the dispenser nor the pump are autoclavable.

TEMCA GmbH & Co. KG

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D-07554 Pölzig

Tel.: +49 3 66 95 322-0

Fax: +49 3 66 95 322-49

info@temca.eu

www.temca.eu