



User Manual

PhysioPortWin

PC-Software for
Long-term blood pressure monitors



PAR Medizintechnik GmbH & Co.KG
Rigistr. 11
12277 Berlin
Germany
www.par-berlin.com

CE 0482

Table of Contents

1	GENERAL INSTRUCTIONS	- 6 -
1.1	ABOUT THIS USER MANUAL.....	- 6 -
1.2	PRODUCT COMPLIANCE	- 6 -
1.3	MARKING	- 7 -
2	INTENDED USE	- 8 -
2.1	INTENDED PURPOSE	- 8 -
2.2	PRODUCT RELEVANT LAWS AND STANDARDS	- 8 -
2.3	USER REQUIREMENTS	- 8 -
2.4	DESCRIPTION OF SOFTWARE	- 9 -
2.5	EXPLANATION OF PARAMETERS	- 9 -
2.5.1	BLOOD PRESSURE	- 9 -
2.5.2	OXYGEN SATURATION	- 10 -
2.5.3	CENTRAL BLOOD PRESSURE	- 10 -
2.5.4	AUGMENTATION PRESSURE AND AUGMENTATION INDEX	- 10 -
2.5.5	PULSE WAVE VELOCITY	- 11 -
3	TECHNICAL DESCRIPTION	- 13 -
3.1	TECHNICAL DATA AND SYSTEM REQUIREMENTS	- 13 -
3.2	ENVIRONMENTAL CONDITIONS FOR THE STORAGE OF THE STORAGE MEDIA	- 13 -
3.3	ACCURACY OF DISPLAY	- 13 -
3.4	INSTALLATION INSTRUCTIONS	- 13 -
3.5	MAINTENANCE REQUIREMENTS	- 14 -
3.6	USING THE SOFTWARE IN NETWORK	- 14 -
3.7	DEPENDENCE ON OTHER APPLICATIONS, GDT INTERFACE	- 15 -
3.8	DISPOSAL	- 15 -
4	INSTALLATION AND START-UP	- 16 -
4.1	INSTALLATION	- 16 -
4.2	STARTING THE PROGRAM	- 18 -
4.3	SETTINGS	- 19 -
4.3.1	SYSTEM	- 19 -
4.3.2	GDT	- 20 -
4.3.3	PRINTING/EXPORT	- 22 -
4.3.4	PARAMETER – BLOOD PRESSURE	- 25 -
4.3.5	PARAMETER SPO2	- 26 -

4.3.6	PARAMETER PWA	- 26 -
4.3.7	SECURITY/AUTHENTICATION	- 26 -
4.4	INSTALLATION IN THE NETWORK	- 28 -
4.5	EXITING THE PROGRAM SAFELY	- 28 -
4.6	UNINSTALLATION	- 29 -
5	START BLOOD PRESSURE MEASUREMENT	- 29 -
5.1	CONNECTING THE RECORDER	- 29 -
5.2	STARTING THE PROGRAM	- 29 -
5.3	SELECT PATIENT	- 30 -
5.3.1	KNOWN PATIENT	- 30 -
5.3.2	CREATE A NEW PATIENT	- 30 -
5.3.3	EDIT PATIENT DATA	- 30 -
5.4	CONFIGURING RECORDER	- 31 -
6	READ IN RECORDER DATA	- 33 -
6.1	CONNECTING THE RECORDER	- 33 -
6.2	START ARCHIVING	- 33 -
7	MEASUREMENT ANALYSIS	- 34 -
7.1	START ANALYSIS	- 34 -
7.2	SEARCH / DELETE ANALYSIS	- 34 -
7.3	ILLUSTRATIONS	- 35 -
7.3.1	SINGLE VALUES	- 35 -
7.3.2	OVERVIEW	- 37 -
7.3.3	HOURLY AVERAGES	- 38 -
7.3.4	COMPARISON	- 38 -
7.3.5	HISTOGRAM	- 39 -
7.3.6	PIE GRAPHIC.....	- 39 -
7.3.7	CORRELATION	- 39 -
7.3.8	FINDINGS AND MEDICATION.....	- 40 -
7.4	EVALUATION OF PULSE WAVE ANALYSES	- 40 -
7.4.1	OVERVIEW OF MEASUREMENT SERIES	- 40 -
7.4.2	DETAILED ANALYSIS OF A PWA MEASUREMENT	- 42 -
8	PWA OFFICE	- 43 -
8.1	PREPARATION	- 43 -
8.2	DESCRIPTION OF THE MEASURING PROCEDURE	- 44 -

9 ADDITIONAL BUTTONS.....	- 46 -
9.1 PRINT	- 46 -
9.2 PDF-EXPORT	- 47 -
9.3 GDT - EXPORT TO PRACTICE-EDP	- 47 -
9.4 CSV-EXPORT	- 48 -

Update history

Update A

B	Date 2017-07-06	Remarks First edition
	2020-04-20	Update of the user manual for product version 1.6 Introduction of new content according to DIN EN 82304-1 for healthcare software Revision of the product labelling (see section 1.3)
C	2022-03-28	Correction of the designation of the tabs in the Settings menu (see Section 4.3.4, 4.3.5 and 4.3.6); correction of Figure 46, as the setting of the measurement intervals for an SpO2 measurement was missing (see Section 5.4); Changing of the manufacturer address
D	2023-05-17	Update for the MDR, updating of Figure 12, updating of the company logo, updating of the installation steps

1 General instructions

1.1 About this user manual

This manual has been prepared with the utmost care. However, if you find details that do not match the system, kindly let us know so that we can resolve any discrepancies as quickly as possible.

We reserve the right to make changes into information and illustrations in these operating instructions due to optical or technical developments.

All trademarks mentioned and depicted in the text are signs of the respective owners and are acknowledged as protected. Reprint, translation and reproduction in any form, even in extracts, require written permission of the manufacturer.

These operating instructions are not subject to any alteration service. For the latest edition, please contact the manufacturer.

The safety instructions in these operating instructions are marked as follows:

Safety advice

Draws attention to hazard or dangerous situation. Failure to do so may result in injury to the patient, the environment and third parties and may cause the PhysioPortWin-Software to fail.



Safety advice

Read the instructions for use before installing and using this software.

1.2 Product compliance

The PhysioPortWin product bears the CE marking "CE 0482" in accordance with the Regulation (EU) 2017/745 of the European Parliament and the Council (MDR) (93/42/EEC (MDD)) and meets the essential safety and performance requirements of Annex I of this Regulation (Directive). It is a medical device software and is assigned to risk class IIa (according to MDR/MDD) and software safety class B (according to IEC 62304). MD code for the software is 1111 and GMDN code: 57967.

The technology developed at PAR Medizintechnik GmbH & Co. KG throughout the company complies with the EN ISO 13485 standard.

Serious incidents that have occurred in connection with the use of this software should be reported to the manufacturer and the competent authority of the member state from which the user and / or patient originates.

1.3 Marking

The following symbols are used on the product and label of the packaging:

Symbols on the product



Follow the instructions.

This symbol refers to safety-relevant information in the operating instructions.



CE Marking.

According to the Regulation (EU) 2017/745 on medical devices
Notified body: MEDCERT GmbH

Manufacturer.



Next to this symbol the company name and address of the manufacturer is indicated.

Manufacturing date.



Next to this symbol, the manufacturing date is displayed in the format YYYY-MM.

Serial Number.



Next to this symbol, the unique manufacturer-specific ten-digit serial number of the product is displayed

WEEE-Symbole indicates electrical and electronic equipment, which must not be disposed of with the normal, unsorted household waste but must be treated separately.



Please contact an authorized representative of the manufacturer to obtain information regarding the disposal of your device.

Symbols on the packaging



Store at dry place.

This symbol indicates that the product must be protected from moisture.



Temperature limitation.

The temperature ranges application or storage are indicated next to this symbol.



Air humidity limitation.

Next to this symbol, the humidity ranges for application or storage are indicated.

Damaged labels must be reported to the manufacturer in order to replace them as soon as possible and ensure that the product can be identified. The information about the manufacturer is on the cover sheet of this manual, information about storage conditions can be found in section 3.2, for disposal in section 3.8. If you have any questions, please contact the customer service of PAR Medizintechnik (service@par-berlin.com).

2 Intended Use

2.1 Intended Purpose

The software is used to set up PHYSIO-PORT and TONOPORT long-term blood pressure monitors and to manage and evaluate the results of such long-term measurements. It enables medical personnel to program the measuring devices to configure a long-term measurement using defined parameters such as the number of measuring intervals, interval between two individual measurements, inflation pressure, etc. Following a long-term measurement, the measurement results stored on the measuring device can be read out by software and stored in the internal software database. Using the database, healthcare professionals can manage any number of patients and measurements. Recorded measurement results can be evaluated according to various criteria at the push of a button. The results of the evaluation are presented in a descriptive form (graphically) and support the physician in the diagnosis of diseases, especially of the cardiovascular system.

The software is only designed for use with NIBP recorders of the PHYSIO-PORT and TONOPORT type. The software is not intended to be used in combination with other NIBP recorders or modules.

The software offers a GDT interface for the connection to external practice software and hospital information systems (HIS). The software can be run on computers in medical practices or hospitals that use a Windows operating system.

The patient group as well as the indication and contraindications depend on the intended purpose of the PHYSIO-PORT and TONOPORT.

2.2 Product relevant laws and standards

- ☒ Regulation (EU) 2017/745 (MDR)
- ☒ Directive 2011/65/EU (RoHS 2) and its amendment (EU) 2015/863 (RoHS 3), (EC) 1907/2006 (REACH)
- ☒ Implementing Regulation as regards electronic instructions for use of medical devices
- ☒ Directive 2019/19/EU together with the standard DIN EN 50419
- ☒ DIN EN ISO 14971 Medical devices – Application of risk management to medical devices
- ☒ DIN EN 62304: Medical device software – Software life-cycle processes
- ☒ DIN EN 82304-1 Healthcare software – Part 1: General requirements for product safety
- ☒ DIN EN ISO 15223-1 Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements

2.3 User requirements

To ensure safe use of the software, it is necessary for users to familiarize themselves with the product by means of these instructions for use.

Apart from a completed medical education, no special skills are required to use the software, as all functions are explained in detail in this document.

2.4 Description of software

The software can also be used to configure approved NIBP recorders (see section 3.1) for long-term blood pressure measurement. Measurement intervals can be configured separately for day and night.

The software reads out the stored data from a NIBP recorder connected to a PC and displays them via this software. The read-in data is stored in the user's system, so that it is also available for later analysis by the user. The user has various display options at his disposal: as a graphical curve of the measured

values during

the measuring interval, as a histogram and as a pie chart. Individual values can be viewed, and minimum and maximum values including time can be examined separately for day and night phases. Furthermore, average values can be calculated over certain time intervals and the correlations between pairs of diastolic, systolic and average blood pressure can be displayed. There is the possibility to document the drug prescription in this program.

The software allows the administration of patient data, which can be converted to PDF-format and then be printed.

A GDT interface allows data exchange with external programs (practice software, hospital information systems HIS), which have a corresponding import function. The software does not interact directly with other programs. The measurement parameters displayed by the software are described in section 2.5.

2.5 Explanation of parameters

The explanations in the following subsections serve to explain the individual parameters, which are clearly displayed with the aid of the software. The blood pressure measured on the upper arm (SYS, DIA, MAP and HR) can be used to assess the cardiovascular condition. The oxygen saturation can be used to assess the supply of oxygen. The three parameter-groups central or aortic blood pressure (cSys, cDia and cPP), augmentation (AugP and AIx) and pulse wave velocity (PWV) are suitable for assessing vascular stiffness or the status of vascular changes.

Note

The limit values given in the following subsections are only intended to assist and guide the user in evaluating the measurement results and are not to be regarded as mandatory requirements for the safe use of the software. The limit values can be freely selected according to user experience. The diagnosis is the sole responsibility of the physician using the software and should not be based solely on the values displayed in the software, but should be made in conjunction with further medical examinations.

2.5.1 Blood pressure

The software displays systolic (SYS), diastolic (DIA) and mean arterial blood pressure (MAP) as well as heart rate (HR). The (higher) systolic blood pressure results when the heart expels blood into the body's arteries; the (lower) diastolic blood pressure results when the pressure in arteries drops during the subsequent filling phase from the heart. In adults, systolic blood pressure should be in the range of

130 mmHg and diastolic blood pressure in the range of 65-85 mmHg¹. High blood pressure is a risk factor for cardiovascular diseases.

2.5.2 Oxygen saturation

The (pulse oximetry measured) oxygen saturation (SpO₂) indicates the proportion of arterial haemoglobin that has absorbed oxygen and is expressed in %.

2.5.3 Central blood pressure

The aortal pulse wave allows conclusions to be drawn about the hemodynamic conditions of the heart. The following three parameters provide information: central systolic blood pressure (cSys), central diastolic blood pressure (cDia) and central pulse pressure (cPP). The central values given correspond to the blood pressure in the ascending aorta. The level of central systolic blood pressure determines afterload on the heart, i.e. the work required for each blood ejection. The level of central diastolic blood pressure is decisive for blood flow in coronary arteries and thus for the supply of oxygen to the heart. The difference between these two pressures is the pulse pressure, which is a measure for assessing degenerative changes in the major arteries. The higher central systolic blood pressure, the greater increase in left ventricular hypertension and the higher central pulse pressure, the worse vascular condition. Table 1 shows typical reference values for central systolic blood pressure (cBPSYS) and central pulse pressure (cPP) depending on patient age and sex.

Table 1: Reference values for central blood pressure in relation to gender and patient age. The mean value \pm one standard deviation is given in each case ²

Age [Years]	Female		Male	
	cBPSYS [mmHg]	cPP [mmHg]	cBPSYS [mmHg]	cPP [mmHg]
< 20	98 \pm 9	25 \pm 6	103 \pm 8	29 \pm 5
20 – 29	101 \pm 9	27 \pm 7	105 \pm 8	20 \pm 6
30 – 39	105 \pm 11	30 \pm 8	109 \pm 9	31 \pm 6
40 – 49	109 \pm 11	33 \pm 8	113 \pm 9	34 \pm 6
50 – 59	115 \pm 11	38 \pm 8	115 \pm 9	35 \pm 7
60 - 69	118 \pm 10	43 \pm 8	117 \pm 9	39 \pm 7
70 – 79	119 \pm 9	56 \pm 8	118 \pm 9	42 \pm 7
\geq 80	120 \pm 11	49 \pm 12	120 \pm 8	45 \pm 9

2.5.4 Augmentation pressure and augmentation index

The increase of blood pressure by pulse wave reflection is called augmentation. The augmentation pressure is the pressure difference of the incoming wave to the pulse pressure caused by the reflected wave. The earlier the reflected wave is superimposed on the incoming wave, the greater the augmentation of the systolic pressure of the incoming wave.

¹ Deutsche Hochdruckliga e.V. DHL. *Patientenleitfaden Bluthochdruck* 2. Auflage 2019; 10

²M. R. Nelson, J. Stepanek, M. Cevette, M. Covalciuc, R. T. Hurst, A. J. Tajik. *Noninvasive Measurement of Central Vascular Pressures with Arterial Tonometry: Clinical Revival of the Pulse Pressure Waveform?* Mayo Clin Proc. 2010;85(5):460-472

The augmentation index indicates the percentage share of the augmentation pressure in the pulse pressure. Augmentation is an indirect parameter of arterial vessel stiffness, but a direct marker of the influence of pulse wave reflection on the aortic blood pressure curve. Augmentation has a characteristic course between the ages of 20 and 50 and is suitable in this age range for assessing vascular condition or cardiovascular risk. Table 2 shows typical reference values for augmentation pressure (AugP) and index (Alx) as a function of patient age and sex. Values that are above the range of the reference values indicate pathologically increased augmentation. In contrast, values that are below range of the reference values specified indicate a good vascular condition.

Table 2: Reference values for augmentation in connection with gender and patient age. The mean value \pm one standard

deviation ³

Age [Years]	Female		male	
	AugP [mmHg]	Alx [%]	AugP [mmHg]	Alx [%]
< 20	1 \pm 3	5 \pm 10	-1 \pm 3	- 2 \pm 8
20 – 29	3 \pm 4	9 \pm 14	1 \pm 4	2 \pm 11
30 – 39	6 \pm 5	20 \pm 12	4 \pm 5	12 \pm 13
40 – 49	10 \pm 5	28 \pm 10	7 \pm 4	19 \pm 10
50 – 59	13 \pm 5	33 \pm 9	9 \pm 5	24 \pm 10
60 - 69	15 \pm 5	34 \pm 9	11 \pm 5	28 \pm 9
70 – 79	16 \pm 5	35 \pm 9	13 \pm 5	30 \pm 9
\geq 80	17 \pm 7	37 \pm 10	14 \pm 5	30 \pm 10

2.5.5 Pulse wave velocity

The contraction of the left ventricle leads to the expulsion of a pressure wave into the vascular system. This pressure wave is called the primary or outward wave. The speed of this wave, the pulse wave velocity, is indirectly proportional to the elasticity of arteries. The lower elasticity, the higher PWV.

The stiffness of vessels increases from the centre to periphery, which means that the velocity in different vessel sections is different. The PWV in a healthy and elastic aorta lies in a range of 4 - 6 m/s at the beginning of life, which shifts to higher values with increasing age. The wave ejected by the heart is reflected in the vascular tree at branches and points where the diameter of the vessel changes. The reflected wave travels back to the heart and is superimposed on the incoming wave. The result is the characteristic pulse waveform.

In the presence of healthy and elastic arteries, the superposition of the reflected wave in the aorta near the heart takes place during diastole. In stiffer vessels, where PWV is elevated, the superposition takes place during systole, changing the pulse waveform. PWV is therefore a direct marker of arterial vessel

stiffness and an independent predictor of a patient's

cardiovascular risk. Compared to augmentation, PWV is suitable for assessing cardiovascular risk from the age of 50. Table 3 shows typical reference values for pulse wave velocity (PWV) as a function of

³ M. R. Nelson, J. Stepanek, M. Cevette, M. Covalciuc, R. T. Hurst, A. J. Tajik. *Noninvasive Measurement of Central Vascular Pressures with Arterial Tonometry: Clinical Revival of the Pulse Pressure Waveform?* Mayo Clin Proc. 2010;85(5):460-472

patient age. Values above the range of the reference values indicate a pathologically increased pulse wave velocity. Furthermore, values of more than 10 m/s represent an additional risk marker for cardiovascular events. In contrast, values below the range of the reference values indicate a particularly good vascular condition.

Table 3: Reference values for pulse wave velocity in relation to patient age ⁴

Age [Years]	Mean value (-2 SD to +2 SD) [m/s]	Median (10. - 90. percentile)
< 30 30	6.2 (4.7 – 7.6)	6.1 (5.3 – 7.1)
– 39 40	6.5 (3.8 – 9.2)	6.4 (5.2 – 8.0)
– 49 50	7.2 (4.6 – 9.8)	6.9 (5.9 – 8.6)
– 59 60	8.3 (4.5 – 12.1)	8.1 (6.3 – 10.0)
– 69 ≥	10.3 (5.5 – 15.0)	9.7 (7.9 – 13.1)
70	10.9 (5.5 – 16.3)	10.6 (8.0 – 14.6)

⁴ P. Boutouyrie et al. *Determinants of pulse wave velocity in healthy people and in the presence of cardiovascular risk factors: 'establishing normal and reference values'*. European Heart Journal 2010;31:2338-2350.

3 Technical description

3.1 Technical data and system requirements

Name:	PhysioPortWin
Version:	v 1.6
Operating System:	Windows XP, Windows Vista, Windows 7, Windows 8 & Windows 10 respectively as 32-Bit-/64-Bit-Version, Windows 11
The software can be used with the following long-term blood pressure measuring devices:	TONOPORT V/VI PHYSIO-PORT PHYSIO-PORT UP PHYSIO-PORT AS (with Pulse wave analysis function) PHYSIO-PORT DUO (with Oxygen saturation measurement function)

Processor:	min. 1.6 GHz Dual Core
Memory:	min. 2 GB
Hard drive capacity:	min. 250 GB
Screen resolution:	min. 1 024 x 768 Pixel
Connectors:	USB (1.1, 2.0 or 3.0)

Vorsicht

• Safety for patients, users and third parties •
Devices may only be connected to each other or to parts of systems if safe coupling is ensured. The PC used must comply with the IEC 60601-1 or IEC 60950-1 standards.

3.2 Environmental conditions for the storage of the storage media

The storage media can be kept under following storage conditions:

- ☒ Temperature: -25 °C ... 70 °C
- ☒ Relative Humidity: 10 % ... 93 %, no condensation
- ☒ Store in dry place.

3.3 Accuracy of display

The PhysioPortWin software graphically displays blood pressure readings recorded by a long-term blood pressure monitor on the computer screen. The accuracy of the display corresponds to the accuracy of the recorded measurement values.

- ☒ Display accuracy blood pressure values: 1 mmHg
- ☒ Display accuracy pulse frequency: 1 bpm
- ☒ Display accuracy oxygen saturation: 1 %
- ☒ Display accuracy Pulse wave velocity: 0.1 m/s

3.4 Installation instructions

The PhysioPortWin software is installed as described in Section 4.1. The installation can be performed by the user (medical personnel). Necessary system libraries are included with the installation. There are no special requirements or restrictions for the installation of this software. If you have any questions regarding installation, please contact the customer service of PAR Medizintechnik (service@par-berlin.com).

3.5 Maintenance requirements

There are no requirements for regular maintenance of the software (e.g. updating). Changes/updates of the operating system, anti-virus software or firewall have no effect on this software. The required system libraries are automatically installed, but not automatically updated. Other system libraries have no effect on this software. In case of problems with the software, please contact the customer service of PAR Medizintechnik (service@par-berlin.com). In case of necessary updates, you will be informed by the manufacturer. The manufacturer explains the update procedure.

3.6 Using the software in network

The software can be used in a network. This network must meet the following requirements:

- ☒ Read and write access to the patient database when two or more workstations share a database
- ☒ The HIS must support the GDT standard (see Section 3.7)

Installation of central database for several workstations

1. To set up a central database, a corresponding data directory must first be defined and released on a central server so that the software has read access to the data stored there and can store new data.
2. Install the PhysioPortWin software locally on the workstations on which the software is to be used.
3. Under [Settings], tab [System], the folder for the shared database is then entered; the integrated Explorer function simplifies the selection.
4. The workstation requires access rights to the central server so that the software can access the data. If access is not granted or the database is not (no longer) available, the error message appears: "Access to the database is not possible". This means that data archiving or analysis of archived data is not possible. The program does not automatically attempt to create a new data directory. The user can set this up on his local workstation, see point 3.

The software does not require the provision of network services. Therefore, changes to the network do not affect the functionality of the Software as long as the Software has access to the database.

Safety advice

• Network risk •

Running this software on a network can lead to previously unknown risks for patients, users and third parties. It is recommended that users identify, analyze, evaluate and manage these risks. Subsequent changes to the IT network may lead to new risks and therefore require a reassessment. Changes to an IT network include, for example, changes to configuration, adding or removing hardware or software components, updating or upgrading hardware and/or software platforms.

Safety advice**• Data protection •**

To prevent data loss, the central server on which the data is stored and the workstation must be protected by up-to-date anti-virus software to ensure protection against malware.

3.7 Dependence on other applications, GDT interface

The software uses standard Windows functions, e.g. for printer output or PDF display. There are no dependencies on third-party programs. It is possible to use the GDT export to make patient data and evaluation results available to other applications that have a GDT import. The data transfer is done via files. There is no direct interaction between the PhysioPortWin software and the importing application of such data. For questions regarding GDR import, it is recommended to consult the manual of the corresponding application. For further questions about the software, please contact PAR Medizintechnik's customer service (service@par-berlin.com).

3.8 Disposal



The storage medium for this software must not be disposed of with the household waste, but separately with the electronic waste disposal. For information on the disposal of the storage medium, please contact the customer service of PAR Medizintechnik (service@par-berlin.com).

WEEE-Reg.-Nr.: DE63208995

4 Installation and start-up

4.1 Installation

To install the PhysioPortWin software, insert the USB stick into a free USB port.

The installation is then started as follows:

- Open the Windows-Explorer
- Select USB stick drive
- Double click on **setup.exe**



The selection menu for the installation language appears (the program language can also be easily changed later in "Settings"):

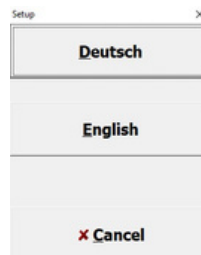


Figure 1: Selection menu Installation language

After selecting the language, the installation will start. The installation is continued via the [Next] button.

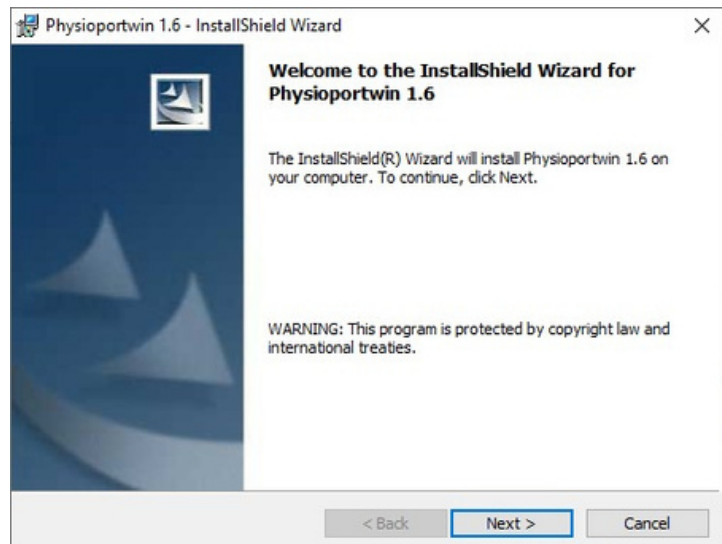


Figure 2: Software Installation

Afterwards the user information is requested and the user can choose whether the software should be installed for all users or only for the currently selected user. The installation is continued via the [Next] button.

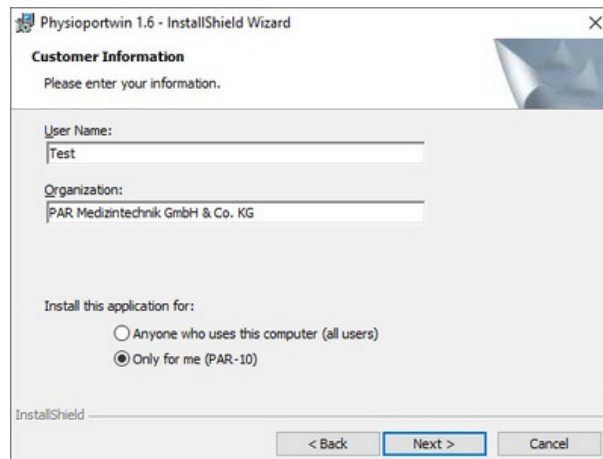


Figure 3: Enter the user information

The default directory for installation is C:\Physioportwin\. By clicking the [Change...] button you can select a different directory.

Note: The drive letter C at the beginning of the directory structure and the directory path are exemplary and can be different for the user.

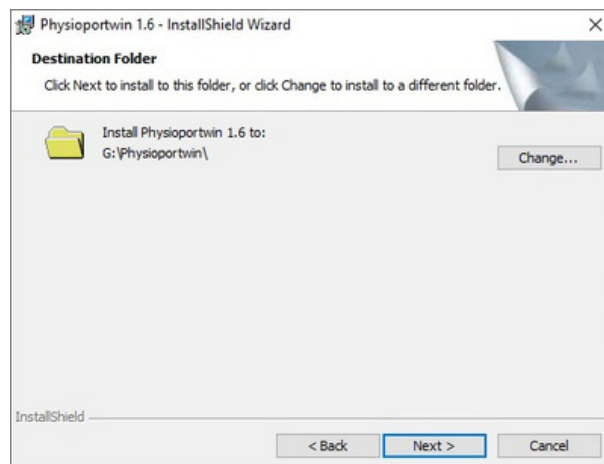


Figure 4: Installation path

Before the actual installation, all information is summarized once again. After confirming the [Install] button, the actual installation process begins.

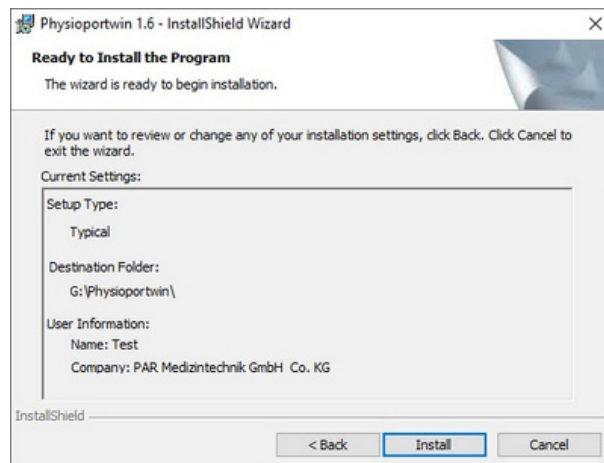


Figure 5: Installation Summary

The installation is completed via the [Finish] button and the installation wizard is closed.

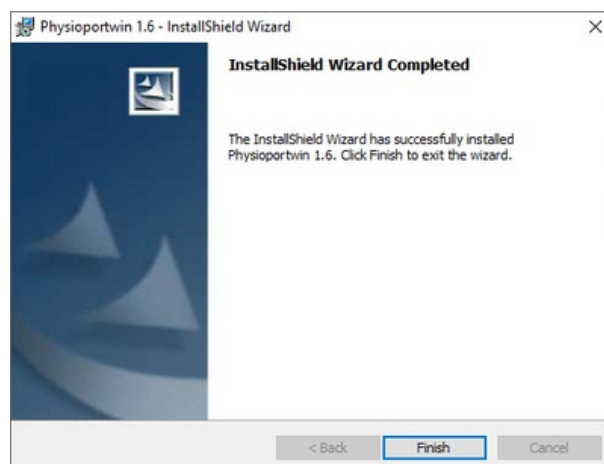


Figure 6: End of installation and program start

4.2 Starting the program

A desktop icon is created during installation (see Figure 7). By double-clicking with the left mouse button on this icon, the program is started.



Figure 7: Desktop icon to start the software

The user is prompted once to enter the serial number (see label on the USB storage device) of the product.

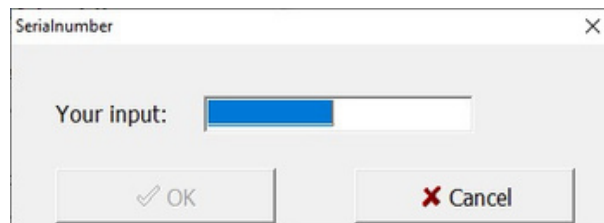


Figure 8: Entering of the serial number

Afterwards, you must log in via the login mask. Enter your user name and password here and confirm your entries with the button [OK]. If you are starting the software for the first time, enter the preset user name and password (see section 4.3.7). Observe the note in the mentioned section on data security.

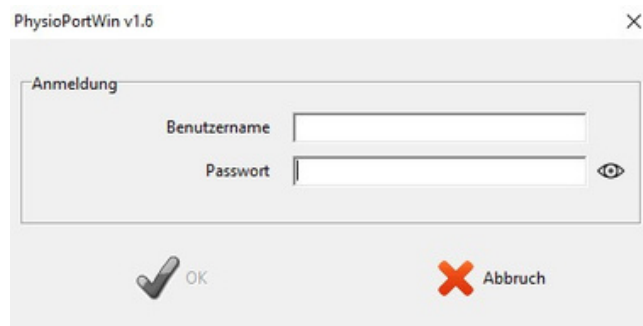


Figure 9: Login mask

The [OK] button can only be selected if the user name and password are entered correctly.

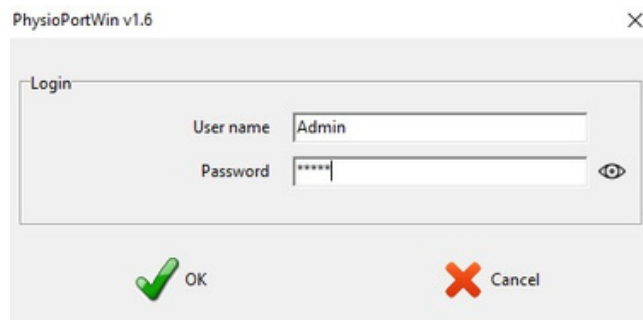


Figure 10: Visualization of a successful login

After correct installation, the main screen appears from Figure 10.

4.3 Settings

After starting the program, the main screen appears. To check the [Settings] page by clicking the corresponding button.

The window contains several options which are explained in the following sections.

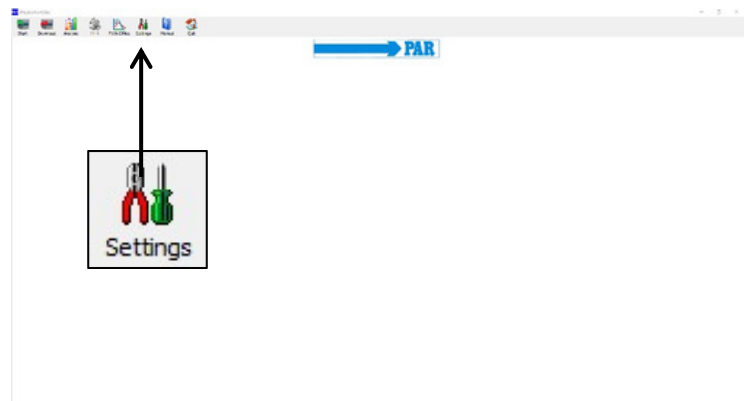


Figure 11: Main menu settings

4.3.1 System

All settings required for standard operation are made on this tab. It is recommended to locate the connected recorder automatically by checking the corresponding setting option, see the figure opposite.

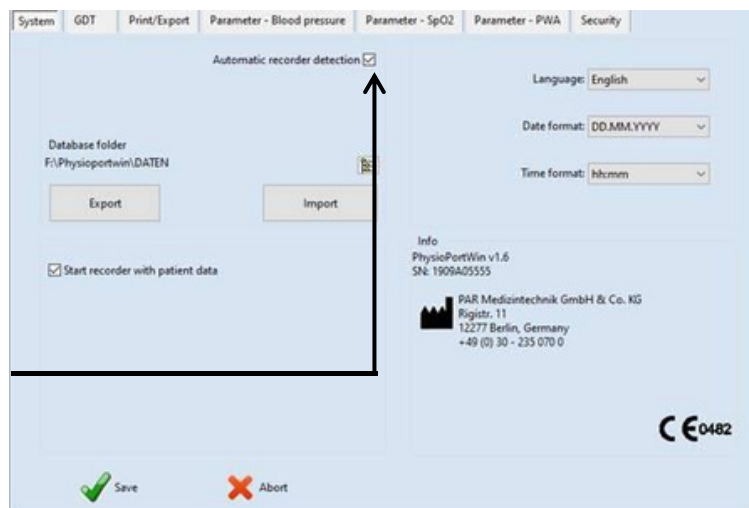


Figure 12: Settings page system

In case of technical problems or when connecting several recorders, the individual interfaces can also be permanently entered. To do this, under "Automatic locate recorder" and then specify the COM port manually.

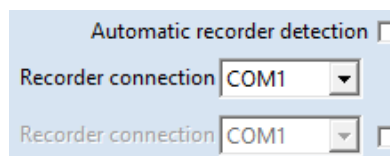


Figure 13: Recorder connection

This setting allows you to [Start] the recorder with or without transferring patient data.

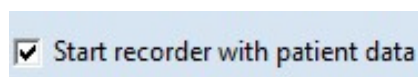


Figure 14: Entering patient data after [Start] the recorder

The folder for the database archiving the data can be set (see Figure 15 and Section 4.4).

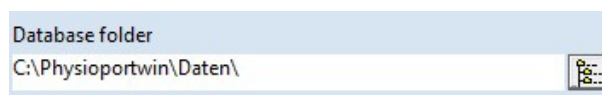


Figure 15: Folder/path for database

The [EXPORT] function saves the database completely in a backup directory. The [IMPORT] function restores a correspondingly backed up database.

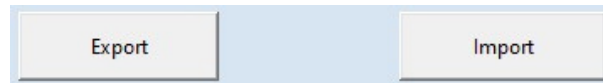


Figure 16: Backup of the database

Caution

•Data loss •

When [Import] a database, the current database is overwritten! The current database should be backed up BEFORE using the [Export] function to avoid data loss!

The program language and the corresponding date and time format can be adapted and set.

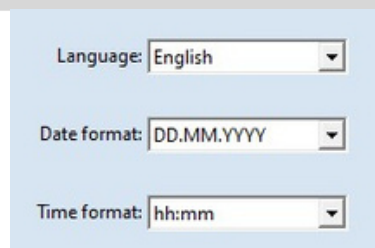


Figure 17: Language, Date and time format

Changes must be confirmed by entering the password when saving (see section 4.3.7).

4.3.2 GDT

On this tab, all settings are made to realize the connection to EDP systems via GDT interface.

The GDT interface (device data transfer) is a software interface mainly used in Germany for the exchange of patient data and evaluation results between practice/clinic EDP systems and medical devices.

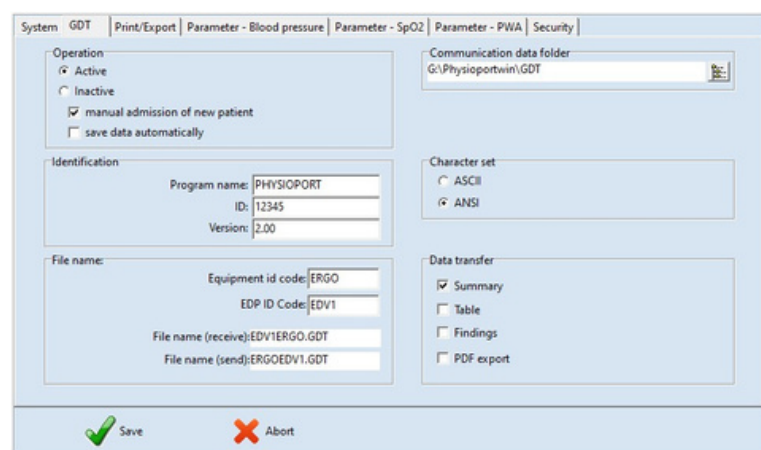


Figure 18: GDT-Settings

Please contact the IT provider which settings of the GDT interface are required for the corresponding IT system. All settings provided in the GDT standard (version 2.0) can be adjusted individually.

Operation: With this setting, the GDT interface can be activated or deactivated. If the GDT interface is activated, the software checks at program start whether the defined GDT file is available and processes it. If no file is found, a "normal" program start takes place.



Figure 19: Activation of the GDT interface

Identification: "ID" is a unique identifier consisting of a minimum of 1 and a maximum of 8 characters that uniquely identifies the PHYSIO-PORT system during GDT data transfer.

Note

In GDT mode, manual patient re-admission should be inactive to avoid incorrect entries and mismatching patient master data (practice EDP and PhysioPortWin database).

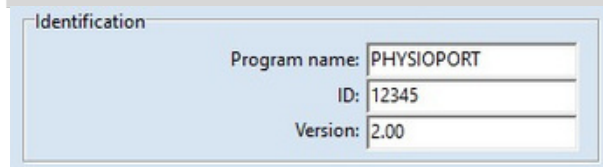


Figure 20: GDT-Identification

File name:

The "File names" field is used to define which file names are used for communication between the practice EDP and PhysioPortWin software.

The file names consist of a device abbreviation (1 - 4 characters), e.g. ERGO, and an EDP abbreviation (1 - 4 characters) for the practice EDP, e.g. EDV1.

The actual file names are composed of these two abbreviations, the file extension is .GDT.

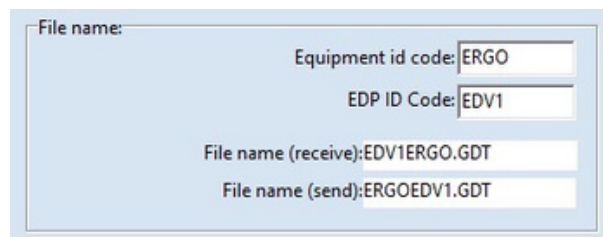


Figure 21: Filename

Directory of the communication files:

The folder for data transfer can be defined as desired. To avoid confusion in networks, a different folder should be set up for each workstation (e.g. on the local PC).

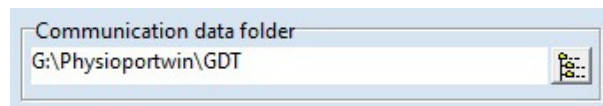


Figure 22: Communication files directory

Note

The GDT file must have been read by the practice EDP and then deleted before another GDT file can be generated.

Font set:

The character set used (ASCII or ANSI) is determined by selecting it accordingly concerns the representation of the letters äöüßÄÖÜ during data transfer.

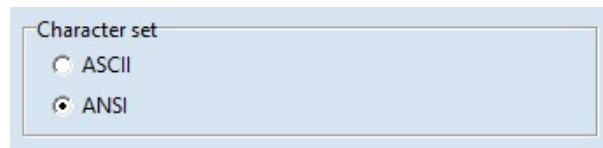


Figure 23: Font selection

Data transmission: An appropriate selection determines which data is transferred to the Praxis-EDP. When the menu item "PDF export" is activated, the GDT file contains a link to the simultaneously created PDF file.

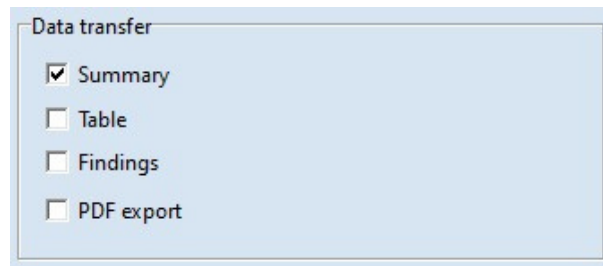


Figure 24: Selection of the data to be transmitted

Changes must be confirmed by entering the password when saving (see section 4.3.7)

4.3.3 Printing/Export

On this tab, all settings concerning the printout of evaluations and the export of data are made.

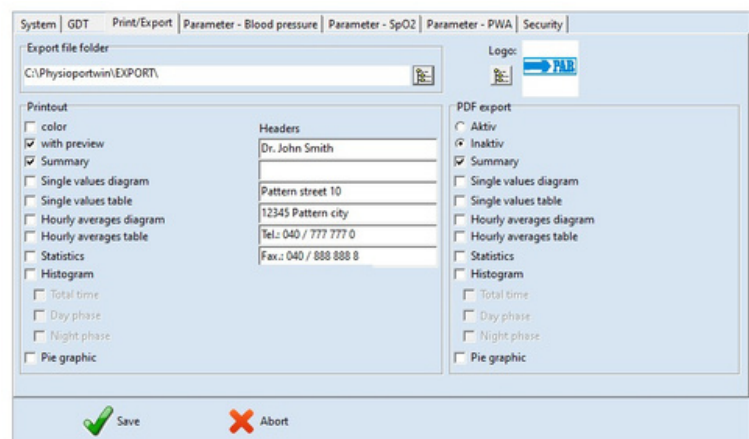


Figure 25: Setting page Printing/ Export

Printout: Defining the standard print pages, print preview and color printout (e.g. for inkjet printers).

The header lines entered (e.g. hospital or practice address) are printed on each page of the printout.



Figure 26: Printing configuration

Additionally, the logo of the clinic or practice can be displayed on the printout. For this purpose, the corresponding logo must be selected.



Figure 27: Logo preview for printing

PDF-Export: Activation of the PDF export and selection of the standard exported pages. When exporting, the remaining pages can be optionally selected.

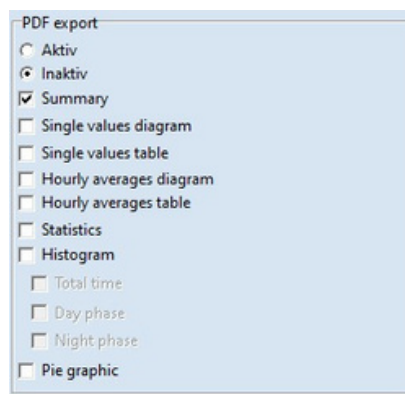


Figure 28: Setting page PDF export

Selection of the folder to save the PDF and CSV files.

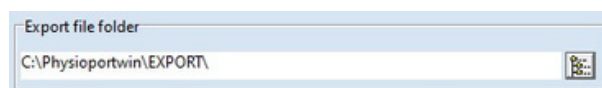


Figure 29: Folder for PDF export files

PhysioPortWin can also generate the printout pages as PDF files (File extension **.PDF**) and the measured values as CSV file (File extension **.TXT**), e.g. for further processing in Excel. The file name contains the following information:

P^<type>_<workstation>^<slot#>_<id>^<last>^<first>_<StartTime>_<exportTime>.**PDF / .TXT**

^ = Block separator

<type> = "REPORT" for blood pressure values, "REPORT-PWA" for pulse wave analysis results or "REPORT-Overview" for the overview of the day/night analysis (see Figure 66)

<workstation> = workstationname

<slot#> = 1 (slotnumber on workstation)

<id> = Patient ID

<last> = Surname

<first> = First name

<StartTime> = Start time of PHYSIO-PORT recording in YYYYMMDDHHMMSS format

<exportTime> = PDF creation time in YYYYMMDDHHMMSS format

z.B.: P^REPORT_Comp^1_12345^Maier^Peter_20031216104632_20040318153145.pdf

P^REPORT-PWA_Comp^1_12345^Maier^Peter_20031216104632_20040318153248.pdf

P^REPORT-Overview_Comp^1_12345^Maier^Peter_20031216104632_20040318153312.pdf

Changes must be confirmed by
entering the password when saving
(see section 4.3.7).

4.3.4 Parameter – Blood pressure

On the tab [Parameter - Blood pressure], you can define the parameters that are used by default when analyzing an evaluation. These parameters can later be changed individually for each individual evaluation. On this tab page, all settings of the parameters for the long-term blood pressure measurement can be made.

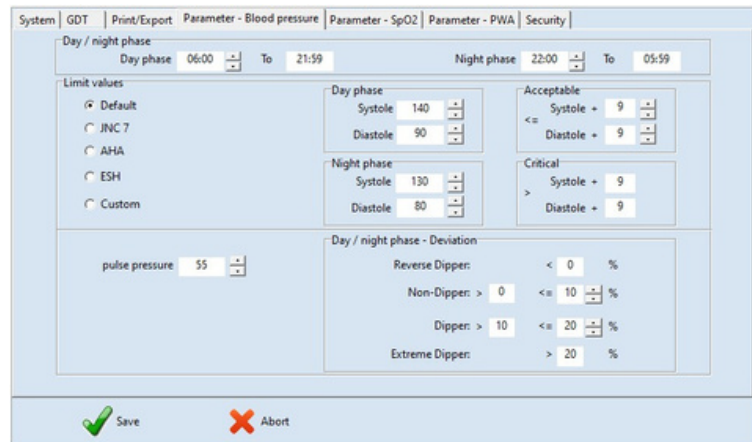


Figure 30: Page settings

Day-/Night Phase:

The standard times for the day phase and night phase can be set.

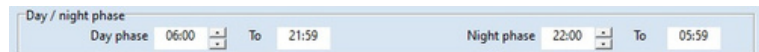


Figure 31: Standard values for day and night phase

Limits:

The values for the limit values of the day and night phase can be defined. These limit values are shown as lines in the graphical representation of the evaluation and are used in the statistical calculations.

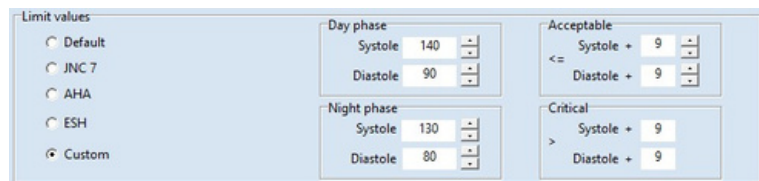


Figure 32: Blood pressure limits

Day/night phase - deviation:

The limits for the nightly pressure drop can be set.

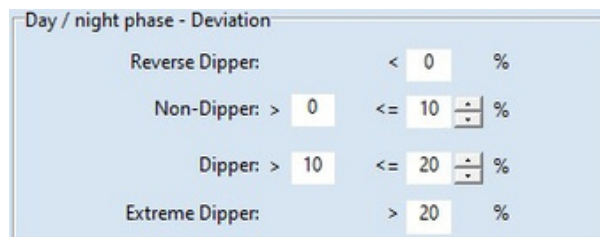


Figure 33: Setting the nightly pressure drop

Changes must be confirmed by entering the password when saving (see section 4.3.7).

4.3.5 Parameter SpO2

On the tab [Parameter - SpO2], you can set the limits for oxygen saturation (SpO2). Changes must be confirmed by entering the password when saving (see section 4.3.7).

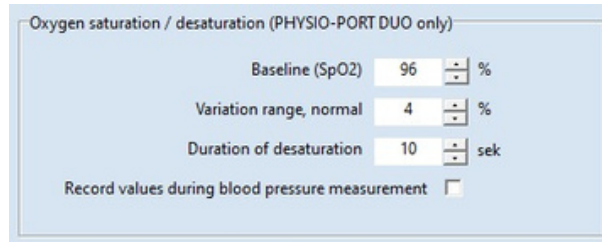


Figure 34: Settings and limit values for oxygen saturation

4.3.6 Parameter PWA

On the tab [Parameter - PWA], the limit values for the parameters of the pulse wave analysis can be set. These values are explained in detail in section 2.5.

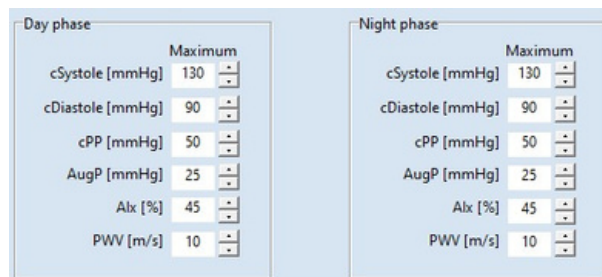


Figure 35: Setting options for PWA parameters.

Notes

The preset limit values must be adapted to the patient in accordance with Section 2.5 in order to enable a patient-specific evaluation of the measured values. The data in the listed tables are only a suggestion and can be freely adjusted by the user according to his experience.

Changes must be confirmed by entering the password when saving (see section 4.3.7).


4.3.7 Security/Authentication

In the [Security] tab, settings for authentication and security can be made.

Notes

The default user name is "Admin", the default password is "Admin". Note that the password is case sensitive.

Via the input field "New user name", the user name can be changed. The password can be changed via the input field "New password". The new password must be repeated in the "Confirmation" input field. The input is encrypted, but by clicking on the

symbol  the password can be displayed as plain text.

In the drop-down list "TimeOut", you can set the time after which the program is locked. You can set 15, 30 or 45 minutes. The current time until the program is locked is displayed.

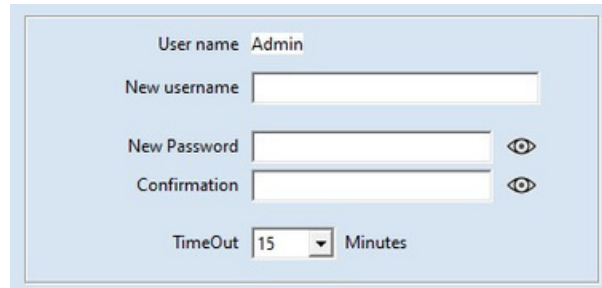


Figure 36: Settings for authentication and security.

After inactivity of mouse keyboard for longer than the specified timeout, the software is locked. To unlock the software, the password used for login must be entered.

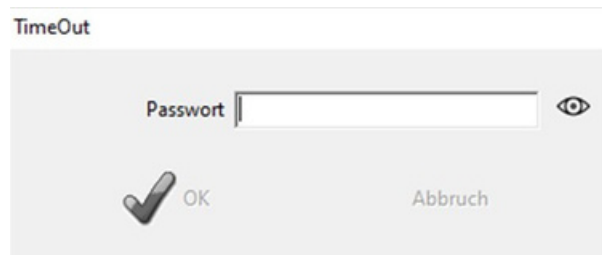


Figure 37: TimeOut Dialog

Each change must be confirmed by entering the password with the old password when saving.

If the password is entered correctly, the [OK] button changes from grey to green, the settings made are active and you return to the main menu.

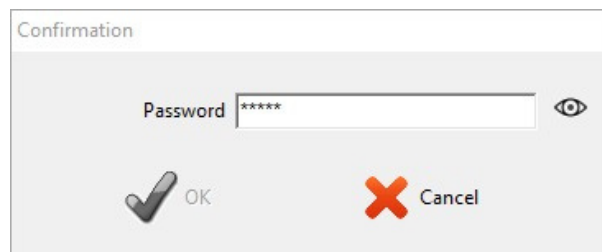


Figure 38: Incorrect password entry

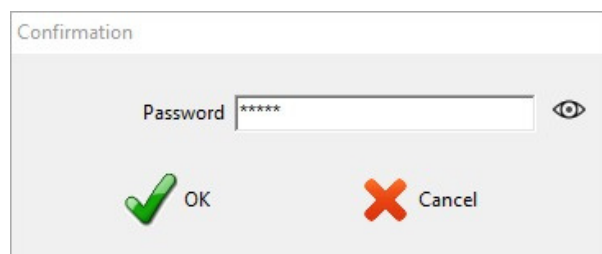


Figure 39: Correct password entry

Safety advice

• Data protection •

Before using the software for the first time, change the default username and password to ensure the protection of health and patient data.

4.4 Installation in the network

The PhysioPortWin software is network-compatible, i.e. patient data and evaluations can be stored centrally (usually on a "server") and this data can be accessed from all workstations.

First, an appropriate data directory is defined and released on a central server.

Then, a local program installation is performed on the workstations on which the PhysioPortWin software is to be used.

Under [Settings], tab [System], the folder for the shared database is then entered, the integrated Explorer function simplifies the selection.

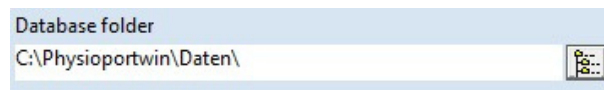


Figure 40: Folder/path for database

4.5 Exiting the program safely

The menu item [Quit] is only available in the main menu. You can return to the previous menu from any submenu via the [?] Cancel] menu area.

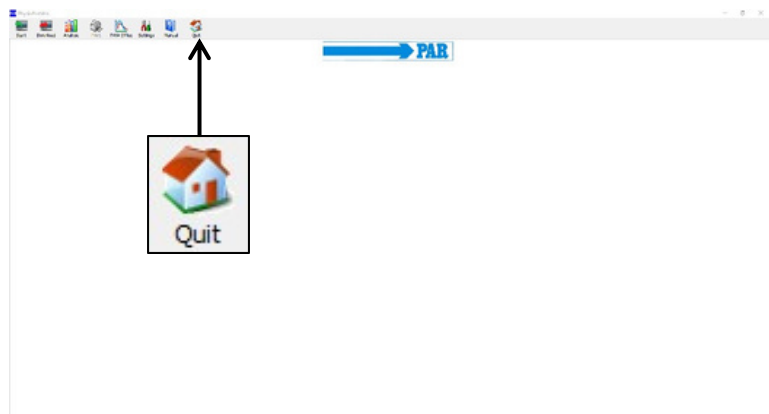


Figure 41: Exit main menu

Additionally, the [X] in the upper right corner of the screen allows you to exit the software from any submenu without returning to the main menu.

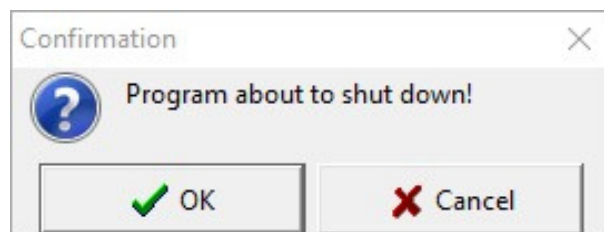


Figure 42: Confirmation dialog for program termination

In both cases, the confirmation prompt appears "The program is terminated!" If you click on the [?] OK] button, the program terminates definitely; if you click on the [?] Cancel] button, the software remains active.

4.6 Uninstallation


The program can be uninstalled as follows:

1. To delete all personal and health-related data, open the database directory (see Section 4.4); delete the "Data" folder and then right-click "Empty Trash" to empty the trash.
2. Call up the Windows control panel: press the key combination [Win] + [R] simultaneously, enter "control" in the input field and confirm with "OK" or [Enter].
3. In the "Uninstall program" menu, find the PhysioPortWin v1.6 software in the list.
4. Right-click on the software to open a window and select the menu item "Uninstall". The software is uninstalled.

5 Start blood pressure measurement

5.1 Connecting the recorder

To configure a long-term measurement, a compatible recording recorder of the PHYSIO-PORT or TONOPORT type (see section 3.1) must be connected to the PC via USB cable and switched on. After the self- test and capacity indication, the display of the recorder shows the symbol for "PC connection". Make sure that the batteries or rechargeable batteries in the unit are fully charged.

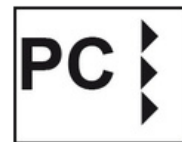

5.2 Starting the program

To configure a recorder, call up the [Start] main menu by clicking the corresponding button.

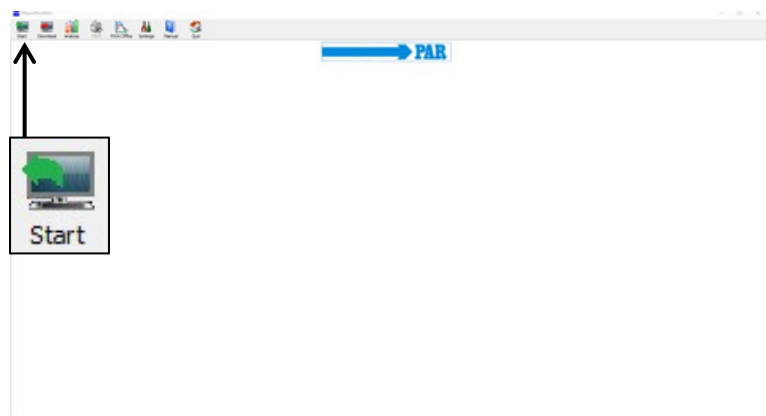



Figure 43: Start main menu

5.3 Select Patient

5.3.1 Known patient

Patient search: In the field "NAME" enter the 1st letter of the last name of the patient you are looking for - all patients whose names begin with this letter (in the example with "M") will appear. The desired patient is selected by double-clicking on the corresponding line.

By means of the buttons [PWA] and [SpO2], the data sets can be filtered according to the parameters.

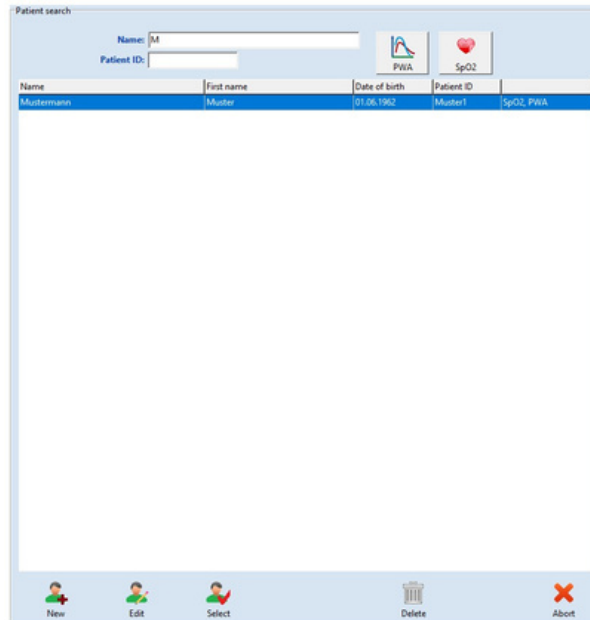


The screenshot shows the 'Patient search' window. At the top, there are input fields for 'Name' (containing 'M') and 'Patient ID'. To the right are two filter buttons: 'PWA' (with a pulse line icon) and 'SpO2' (with a heart icon). Below these is a table with the following columns: Name, First name, Date of birth, Patient ID, and SpO2, PWA. The table contains one entry: 'Mustermann', 'Muster', '01.06.1982', 'Muster1', and 'SpO2, PWA'. At the bottom of the window are five buttons: 'New' (with a plus icon), 'Edit' (with a pencil icon), 'Select' (with a checkmark icon), 'Delete' (with a trash can icon), and 'Abort' (with a red X icon).

Figure 44: Patient selection screen, all patients with "M"

5.3.2 Create a new patient

If the patient you are looking for is not yet in the database, the new patient data can be entered directly. To do this, open the input mask with the [NEW] button, enter all necessary data and save them in the database with [Save]. If required, the current medication can be entered by the patient.

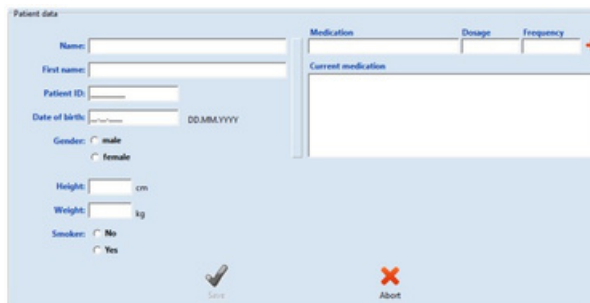

The screenshot shows the 'Patient data' window. It has several input fields: 'Name', 'First name', 'Patient ID', 'Date of birth' (with a date picker icon), 'Gender' (radio buttons for 'male' and 'female'), 'Height' (with a unit 'cm'), 'Weight' (with a unit 'kg'), and 'Smoker' (radio buttons for 'No' and 'Yes'). To the right, there is a section for 'Medication' with sub-headers 'Medication', 'Dosage', and 'Frequency', and a '+' button. Below this is a table for 'Current medication'. At the bottom are two buttons: 'Save' (with a checkmark icon) and 'Abort' (with a red X icon).

Figure 45: Patient new admission

5.3.3 Edit patient data

Once patient data has been entered, it can be edited and changed at any time using the [Edit] button in the patient search mask.

Note

The patient ID cannot be changed after it has been entered and saved because it is unique.

5.4 Configuring recorder

After selecting the desired patient, the screen for configuration / programming the recorder for long-term measurement appears.

Only for recorders with SpO2 function, the "Measurement" area appears, which can be used to determine whether only blood pressure, only SpO2 or both parameters should be determined.

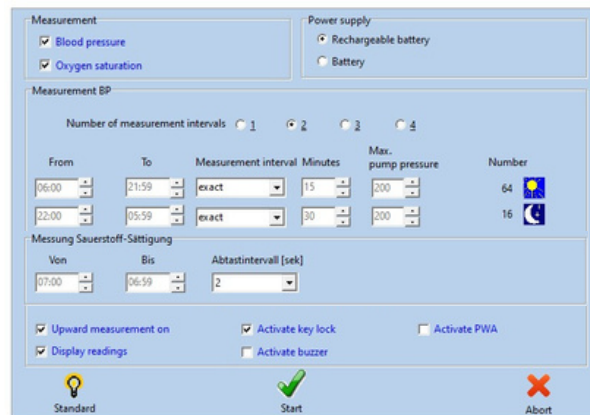


Figure 46: Measurement parameters

Number of measuring intervals: Up to 4 different measuring intervals can be set, two intervals (day and night phase) are offered as standard. For each measuring interval, the interval, measuring distance, minutes and maximum pump pressure can be set individually.



Figure 47: Determination of the number of measuring intervals

Power supply:

It is determined with which type (battery or rechargeable battery) the current long-term measurement is performed.

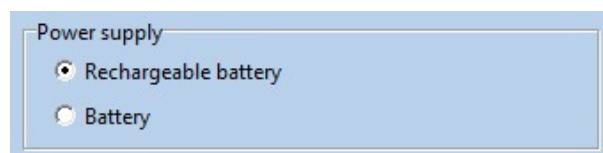


Figure 48: Determining power supply

Interval duration (From ... To ...):

Start and end of the respective interval can be set by the hour. The measuring intervals can be set separately for recorders with SpO2 function.

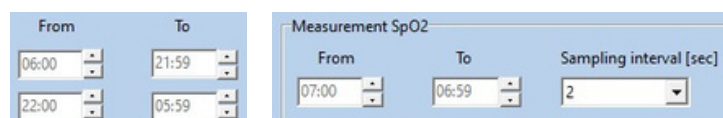


Figure 49: Time window for the individual blood pressure measurement intervals (left) and for SpO2 measurement (right)

Measuring distance:

With the "exact" setting, measurements are taken exactly at the set minute intervals. With the "approx." setting, the measurement intervals vary randomly by ± 2 minutes by the set time.

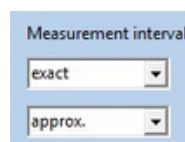


Figure 50: Setting the accuracy of the measuring distance

Measuring interval:

The measuring interval between measurements can be configured between 2 and 120 minutes.

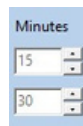


Figure 51: Setting the time interval between measurements

Max. Pump pressure:

Limits the maximum cuff pressure to the set value between 200 mmHg and 280 mmHg.

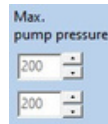


Figure 52: Setting the maximum cuff pressure during a blood pressure measurement

Quantity:

The total number of measurements is displayed both for each individual measuring interval and for 24 hours.



Figure 53: Number of measurements per interval

During 24 hours 80 measurements are taken.

Figure 54: Number of measurements during long-term measurement

☐ Display readings

The blood pressure values are shown on the display of the recorder.

☐ Day-/Night switch activated

If ticked the day-/night switch of the recorder is activated.

☐ Upward measurement on

The blood pressure measurement is performed with the upward measurement method (Inflation Measurement Technology)

☐ Activate buzzer

The internal beep (optional) can be switched on and off. If the beep is active, an acoustic signal sounds after switching on, before the measurement and in case of an incorrect measurement.

☐ Activate PWA

Activates the pulse wave analysis function if the monitor supports it. If this function is activated, a pulse wave analysis is performed after each blood pressure measurement.

Standard:

The parameter button to reset the configuration is shown in Figure 46.

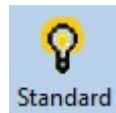


Figure 55: Button to reset the performed configuration

After configuration, press the [Start] button to start the recorder.

First, it checked whether there are any measurements in the recorder that have not yet been read in - if so, a corresponding warning is issued. If not, the memory of the recorder is cleared and the new patient data and measurement intervals are transferred to the recorder.

CAUTION

• Patient confusion •

The recorder must be restarted by the PhysioPortWin software before each new measurement. Otherwise, new measurements are assigned to the old patient (who is still stored in the recorder). Only when a new long-term measurement is started will the old patient data and measurement values be deleted from the recorder.

6 Read in recorder data

6.1 Connecting the recorder

To read in and then archive the measurement data from the recorder, connect the recorder to the PC according to section 5.1.

6.2 Start archiving

The archiving of measurement data can be started with the [Download] button in the main menu.

The measured values are read out from the recorder and stored in the database for the corresponding patient.

Then the graphical evaluation of the results is displayed, which is explained in section 7.

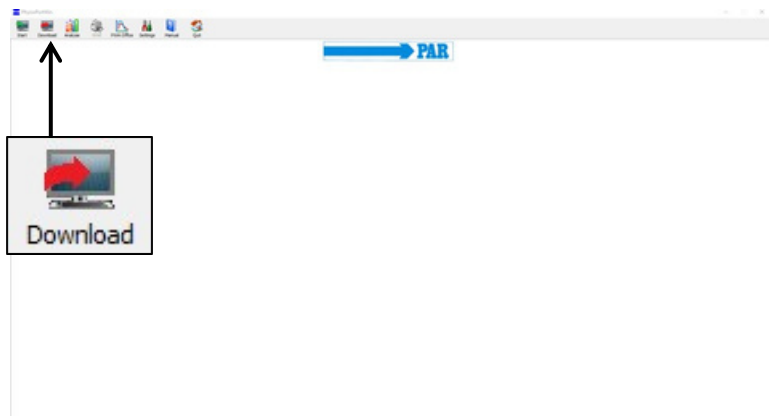


Figure 56: Read main menu

If data is received incorrectly from the connected device, an error message appears and the read-in process must be restarted after acknowledging the message.

Note

All available data is always read in from the recorder during reading.

7 Measurement analysis

7.1 Start analysis

The [Analyse] button is used to call up and display stored analysis.

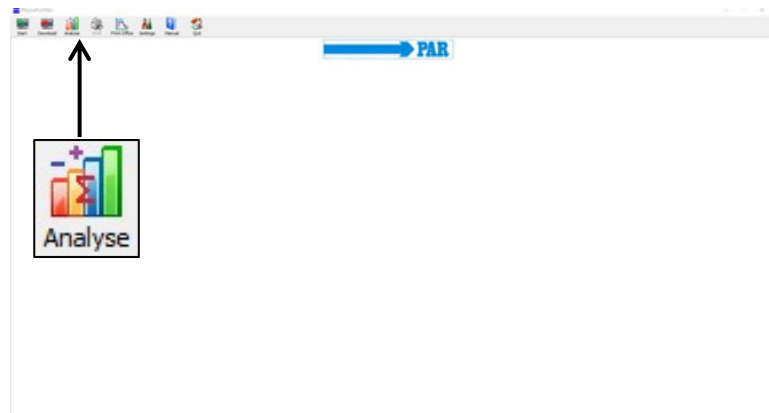


Figure 57: Main menu analysis


Search / Delete analysis

The selection mask of the patient database appears and the patient can be selected according to section 5.3.1. The screen for [Select] the analysis that is stored in the database for the selected patient appears.

In the selection window, the saved analysis is called up by double-clicking on the corresponding date or by marking and clicking on the [Select] button.

To delete a specific evaluation from

the database, select the corresponding date line in selection mask (line appears in blue) and then press the [Delete] button. This evaluation is deleted after a corresponding query.

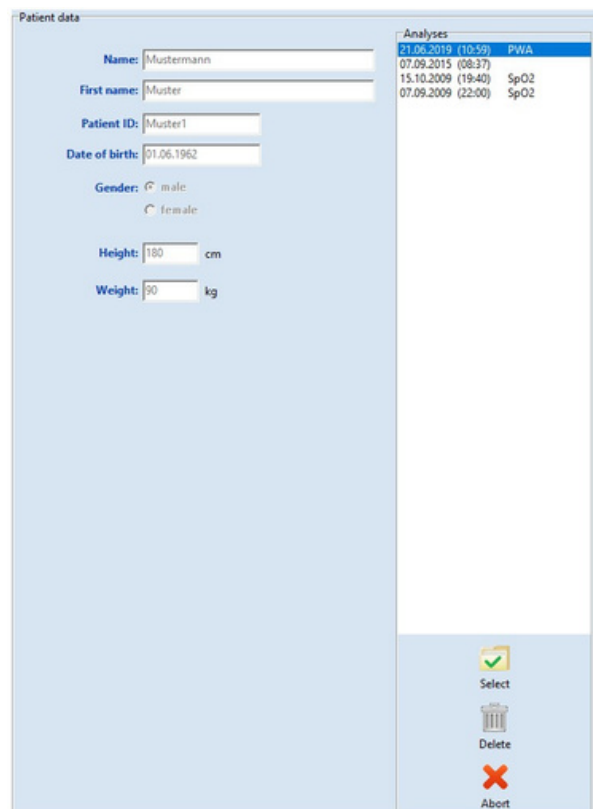

Analyses		
21.06.2019	(10:59)	PWA
07.09.2015	(08:37)	
15.10.2009	(19:40)	SpO2
07.09.2009	(22:00)	SpO2

Figure 58: Analysis selection mask

Warning

• Loss of data •

Make a regular backup of your database, including individual records, so that you can restore data in the event of database or record corruption. Customer service can help you with the recovery.

If a record in the database is damaged, an error message appears and the record can no longer be analyzed.

7.3 Illustrations

The other display formats can be called up by clicking on the corresponding index card.

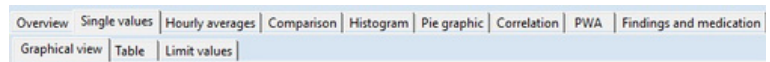


Figure 59: Choose display

7.3.1 Single values

Graphical view: After activating a specific evaluation, the stored measured values are first displayed graphically. The diagram shows the results of all individual measurements. The set limit values for day and night phase (see Settings) are shown as red lines. By moving the cursor over the

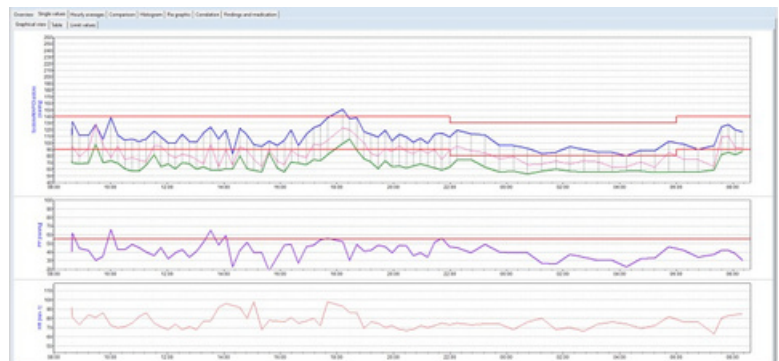


Figure 60: Graphic of the single values

corresponding measured curves, each individual measured value is displayed.

During a PHYSIO-PORT measurement, a graphic of the SpO2 values is also displayed.

Magnifying glass function: In the [Graphical view] tab of the [Single values] tab, the magnifying glass function can be used. The integrated magnifying glass function is used to display the measured values more precisely.

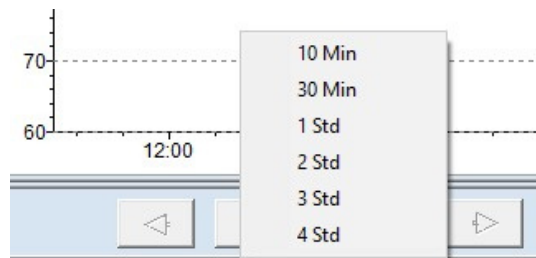


Figure 61: Selectable time windows for the magnifying glass function

The magnifying glass increases the temporal resolution of the displayed section and can be shifted by an adjustable range of 10 min - 4h. The time window can be selected by clicking on the down arrow. The arrows to the left and right are used to shift the time window.

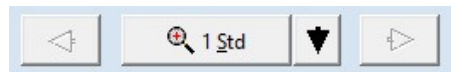


Figure 62: Magnification function

By clicking on the time interval or by clicking in the graphics the magnifying glass is activated. A further click in the graph or on the button deactivates the magnifying glass and shows the complete evaluation again.

Table:

In the tabular representation, measurement results are listed with date / time, systole, diastole, heart rate and mean pressure.

Additional measurements that were triggered manually via the start-stop button are marked with "+" sign after the time, values from the night phase are marked by "*" and values from an upward measurement are highlighted by "^".

Overview: Single values Hourly averages Comparison Histogram Pie graphics Correlation Findings and medication						
Graphical view: Table		Limit values				
Time		Systole (mmHg)	Diastole (mmHg)	MAP (mmHg)	HR (min ⁻¹)	Notes
07.08.2015 08:07		112	70	80	80	
07.08.2015 08:08+		112	70	84	81	
07.08.2015 08:53		112	68	79	73	
07.08.2015 08:13		111	69	81	84	
07.08.2015 08:28		128	98	129	81	
07.08.2015 08:43		105	70	87	86	
07.08.2015 10:00		139	73	78	72	
07.08.2015 10:15		112	69	85	70	
07.08.2015 10:30		104	61	75	71	
07.08.2015 10:45		106	57	76	73	
07.08.2015 11:00		102	57	74	82	
07.08.2015 11:15		104	60	71	86	
07.08.2015 11:32		118	82	96	73	
07.08.2015 11:47		108	64	85	71	
07.08.2015 12:02		100	68	83	69	
07.08.2015 12:17		100	61	77	74	
07.08.2015 12:32		113	70	83	68	
07.08.2015 12:47		102	68	80	71	
07.08.2015 13:02		101	60	75	68	
07.08.2015 13:17		115	64	68	77	
07.08.2015 13:32		124	59	98	77	
07.08.2015 13:49		106	58	63	82	
07.08.2015 14:04		120	61	80	86	
07.08.2015 14:19		83	60	66	84	
07.08.2015 14:34		122	80	94	82	
07.08.2015 14:49		112	61	89	80	
07.08.2015 15:04		97	58	75	88	
07.08.2015 15:21		95	56	64	88	
07.08.2015 15:36		103	83	90	78	
07.08.2015 15:53		96	62	72	77	
07.08.2015 16:08		104	59	67	76	

Figure 63: Table of individual values

Note

- Deleting individual measurements from a data set •

To delete individual measurements, activate the line in the table with a single mouse click (line appears in blue) and then press the [Delete] button. After a corresponding query, this measurement is deleted.

Limit values: The limit values, day/night phase, tabular and graphic individual values are displayed in summary form. Both the limit values and the day/night phase can be set individually here. For the limit values, the following settings are available as described in Section 4.3.4.



Figure 64: Display and setting of individual values, day/night phase and limit values

7.3.2 Overview

The [Overview] shows the numerical summary of the long-term blood pressure measurement / (SpO2 monitoring optionally below) and statistical evaluations both for the entire monitoring period and separately for day and night phase.

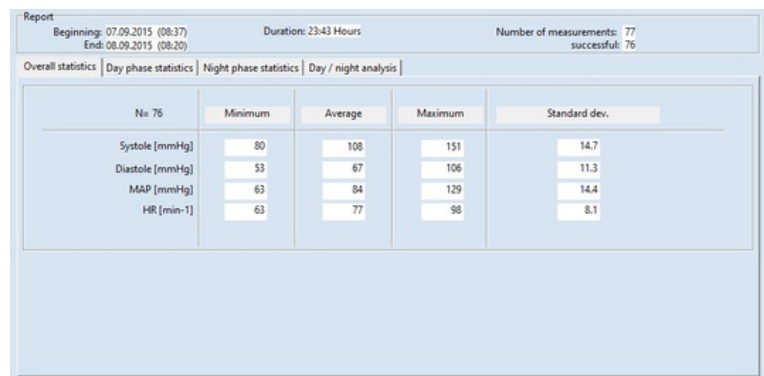


Figure 65: Overall statistics of a long-term measurement

The software calculates Dipper, Non-Dipper, Extreme Dipper or Reverse Dipper values in the overview of the day/night analysis.

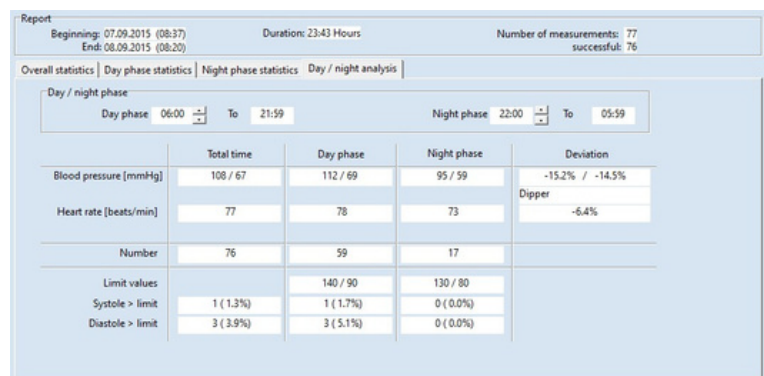


Figure 66: View of a day/night analysis with information about dipper type of patient

7.3.3 Hourly averages

The hourly mean values of all clinically significant, measured parameters including pulse pressure (difference between systole and diastole) and PRP ((systole times heart rate)/1000) are displayed in the tab of the same name. A graphic display is also available (see [Graphic] tab page).

Overview Single values Hourly averages Comparison Histogram Pie graphic Correlation Findings and medication							
Graphical view Table							
Time	Systole [mmHg]	Diastole [mmHg]	MAP [mmHg]	HR [min-1]	PD [mmHg]	PRP / 1000	Number
07.09.2015 08:00 - 09:00	118.7	70.0	88.3	82.0	48.7	9.7	3
07.09.2015 09:00 - 10:00	114.7	79.0	105.7	83.7	35.7	9.6	3
07.09.2015 10:00 - 11:00	115.3	65.0	81.5	72.0	50.3	8.3	4
07.09.2015 11:00 - 12:00	108.8	67.3	84.0	78.5	41.5	8.5	4
07.09.2015 12:00 - 13:00	103.8	66.8	80.8	70.3	37.0	7.3	4
07.09.2015 13:00 - 14:00	111.5	60.3	76.0	78.5	51.3	8.8	4
07.09.2015 14:00 - 15:00	109.3	65.5	84.8	90.5	43.8	9.9	4
07.09.2015 15:00 - 16:00	97.8	65.3	75.3	80.3	32.5	7.8	4
07.09.2015 16:00 - 17:00	108.3	65.8	78.8	77.3	42.5	8.4	4
07.09.2015 17:00 - 18:00	129.3	76.7	99.3	83.3	52.7	10.8	3
07.09.2015 18:00 - 19:00	135.5	92.5	113.3	83.5	43.0	11.3	4
07.09.2015 19:00 - 20:00	111.0	67.3	85.5	73.3	43.8	8.1	4
07.09.2015 20:00 - 21:00	107.5	65.0	89.3	68.8	42.5	7.4	4
07.09.2015 21:00 - 22:00	109.0	62.3	84.5	72.5	46.8	7.9	4
07.09.2015 22:00 - 23:00	116.5	74.5	91.5	74.0	42.0	8.6	2
07.09.2015 23:00 - 00:00	104.0	59.5	80.5	74.0	44.5	7.7	2
08.09.2015 00:00 - 01:00	94.0	55.0	73.0	72.0	39.0	6.8	2
08.09.2015 01:00 - 02:00	85.0	58.5	70.0	74.0	26.5	6.3	2
08.09.2015 02:00 - 03:00	92.0	56.5	70.5	68.0	35.5	6.3	2
08.09.2015 03:00 - 04:00	86.0	56.0	67.0	75.0	30.0	6.5	2
08.09.2015 04:00 - 05:00	84.5	57.0	67.5	71.5	27.5	6.0	2
08.09.2015 05:00 - 06:00	95.0	55.5	74.5	77.0	39.5	7.3	2
08.09.2015 06:00 - 07:00	94.0	56.0	75.0	76.0	38.0	7.1	2
08.09.2015 07:00 - 08:00	116.0	75.7	94.0	75.3	40.3	8.7	3
08.09.2015 08:00 - 09:00	118.5	84.5	92.5	84.5	34.0	10.0	2

Figure 67: Tabular representation of the hourly average values

7.3.4 Comparison

A comparison of two measurement series both in graphical and tabular form is possible in the [Comparison] tab. On the left side of the display, the

currently selected series is shown. A measurement series can be selected for comparison using the "Date" dropdown menu on the right-hand side.

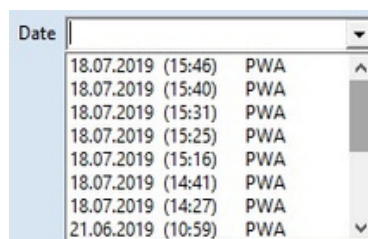


Figure 68: Dropdown to select available measurement series for comparison

Date	07.09.2015 (08:37)	18.07.2019 (15:46) PWA
Blood pressure [mmHg]		
Total time	108 / 67	118 / 78
Day phase	112 / 69	118 / 78
Night phase		
Deviation		
Heart rate [beats/min]		
Total time	77	80
Day phase	78	80
Night phase		
Deviation		

Figure 69: Tabular comparison of two measurement series of a patient

7.3.5 Histogram

The frequency of certain blood pressure values/value ranges as well as the heart rate during the day (see tab [Day phase]), at night (see tab [Night phase]) and in the entire measuring interval (see tab [Total time]) can be represented at a glance using the histogram display.



Figure 70: Histogram display of the total measuring time

7.3.6 Pie graphic

The measured values can also be displayed as a pie chart, so that the proportion of critical measured values that exceed the set limit value can be visually recorded.



Figure 71: Display of the measured values as a pie chart

7.3.7 Correlation

the the [Correlation] tab, correlation of the blood parameters can be displayed. A combination of two blood pressure parameters can be selected via a drop-down menu.

The diagram shows the correlation between diastolic and systolic blood pressure. Each point corresponds to one measurement.

In addition, the relationship between blood pressure values and mean arterial pressure can be displayed.

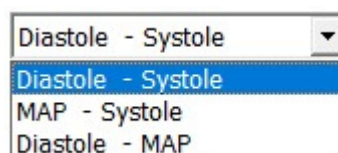


Figure 72: Dropdown for selecting the parameters for correlation display

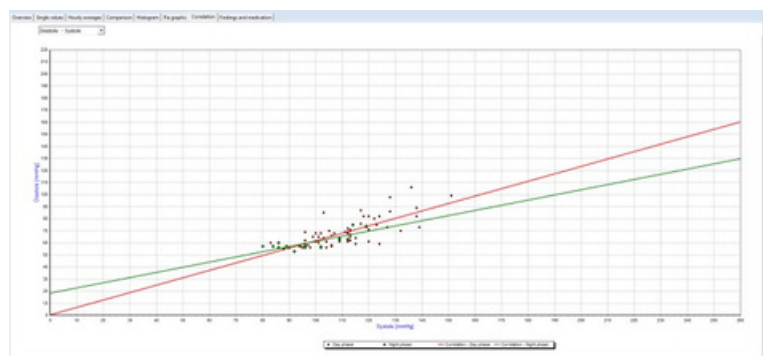


Figure 73: Correlation of systolic and diastolic blood pressure values

7.3.8 Findings and medication

In the tab [Findings and Medications], the user can enter and save a finding for the current measurement series.

This text is also printed on the summary page and can be transferred to the practice EDP.

In addition, the user can view the current medication if he has inserted it in the patient file and adjust it if necessary.

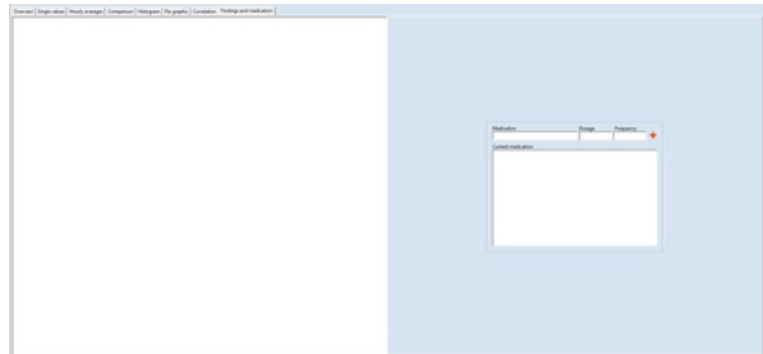


Figure 74: Freely editable findings and medication list

7.4 Evaluation of pulse wave analyses

7.4.1 Overview of Measurement series

Start the analysis via the [analysis] button (see section 7.1). Via the button [PWA], a patient filtering takes place and only patients with pulse wave analyses are displayed. The patient search is described in section 5.3.1. Then select the measurement series (see Section 7.2).

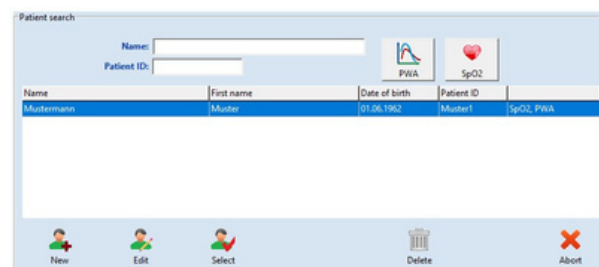


Figure 75: Patient search

Note

Measurements without abbreviations contain blood pressure readings. The abbreviation SpO2 indicates that the measurement contains oxygen saturation data. The abbreviation PWA indicates that the measurement contains pulse wave analysis data.

Click the [PWA] tab (1) to view the results of the selected test.

The navigation area on the left side

lists all measurements of the measurement series. By clicking on the date of the examination (2.1), an overview of all measurements of the examination is displayed on the right side and you get to the overview of all measurement ranges. By clicking on the time of a measurement (2.2), the results of this single measurement are displayed as described in section 7.4.2.



Figure 76: Complete overview of PWA measurement series

Overview:

This tab shows an overview of the entire measurement series.

The measurements, of minimum, average, maximum and standard deviation of the peripheral and central blood pressure and the parameters of the pulse wave analysis (vascular system) of the measurement series are displayed.

the the "Graphics" column, temporal course of each parameter can be displayed individually.

The set limit values (see section 4.3.6) are shown as red lines in the graphic. The limit values can be changed directly via the [Settings] button.



Figure 77: Complete overview of the PWA data of a measurement series

Single values: This display can be reached by clicking on the [Single values] tab. The time of measurement and the measured values of central systolic blood pressure (cSystole) and central diastolic blood pressure (cDiastole),

central pulse pressure (cPP), augmentation pressure (AugP) and pulse transit time (PTT) and pulse wave velocity (PWV) are listed in a table.

Overview	Single values							
Time	cSystole [mmHg]	cDiastole [mmHg]	cPP [mmHg]	AugP [mmHg]	Abz [%]	PTT [ms]	PWV [m/s]	
24.03.2018 16:15	105	78	27	16	58	102	8.2	
24.03.2018 16:21	105	78	27	16	60	102	8.2	
24.03.2018 16:30	105	78	27	16	60	102	8.2	
24.03.2018 16:45	105	78	27	16	60	102	8.2	
24.03.2018 17:00	105	78	27	16	60	102	8.2	
24.03.2018 17:15	105	78	27	16	59	102	8.2	
24.03.2018 17:30	105	78	27	16	60	102	8.2	
24.03.2018 17:45	105	78	27	16	59	102	8.2	
24.03.2018 18:00	105	78	27	16	59	102	8.2	
24.03.2018 18:15	105	78	27	16	60	102	8.2	
24.03.2018 18:30	105	78	27	15	58	102	8.1	
24.03.2018 18:45	105	78	27	16	59	102	8.2	
24.03.2018 19:00	106	79	27	16	60	102	8.2	
24.03.2018 19:15	105	78	27	16	59	102	8.2	
24.03.2018 19:30	105	78	27	16	59	102	8.2	
24.03.2018 19:45	105	78	27	16	60	102	8.2	
24.03.2018 20:00	105	79	26	15	59	102	8.1	
24.03.2018 20:15	105	78	27	16	59	102	8.2	
24.03.2018 20:30	105	78	27	16	59	102	8.2	
24.03.2018 20:45	105	78	27	16	60	102	8.2	
24.03.2018 21:00	106	79	27	16	60	102	8.2	
24.03.2018 21:15	105	78	27	16	60	102	8.2	
24.03.2018 21:30	105	78	27	16	60	102	8.2	
24.03.2018 21:45	106	79	27	16	60	102	8.2	
24.03.2018 22:00	106	79	27	16	60	102	8.2	
24.03.2018 22:30	105	78	27	16	60	102	8.2	
24.03.2018 23:00	105	79	26	16	60	102	8.1	
24.03.2018 23:30	105	78	27	16	60	102	8.2	
24.03.2018 00:00	105	78	27	16	59	102	8.2	
24.03.2018 00:30	105	78	27	16	60	102	8.2	
24.03.2018 01:00	105	78	27	16	60	102	8.2	
24.03.2018 01:30	105	78	27	16	60	102	8.2	

Figure 78: Single value table of the PWA data of a measurement series

7.4.2 Detailed analysis of a PWA measurement

Overview: The patient data (top left), a graphic representation of the peripheral pulse wave (bottom left) and the results of the displayed measurement are marked in red and are not taken into account in the analysis.

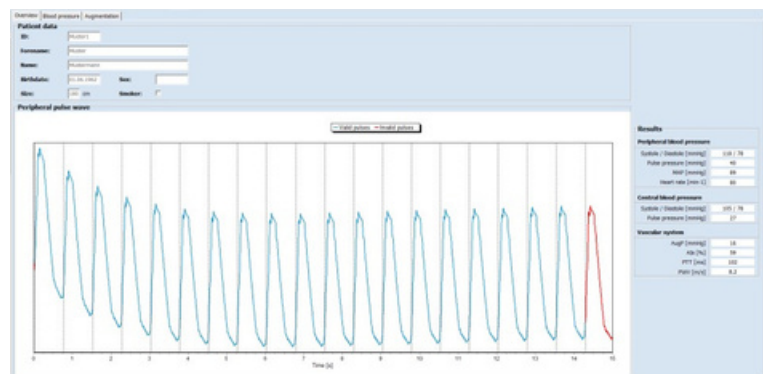


Figure 79: Overview of the PWA data of a single measurement

Blood pressure:

The [Blood pressure] tab displays the mean peripheral and the reconstructed aortal pulse wave. You also receive a display of the blood pressure values determined.

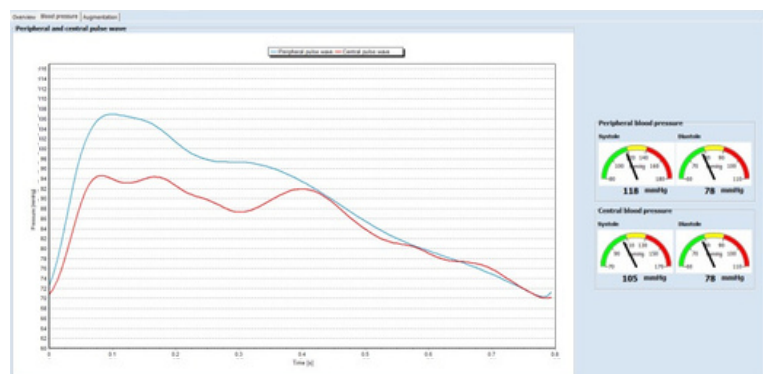


Figure 80: Display of the blood pressure values

Augmentation: The [Augmentation] tab shows the decomposition of the aortic wave into its reflected propagating and components. These waves are used to determine the augmentation. Points P1 and P2 for determining the augmentation parameters are shown in the Pulse wave separation action area.

On the right, the augmentation pressure (AugP), which is calculated from the difference between P1 and P2, and the augmentation index (AIx), which indicates the percentage of the augmentation pressure in relation to the pulse pressure, are displayed.

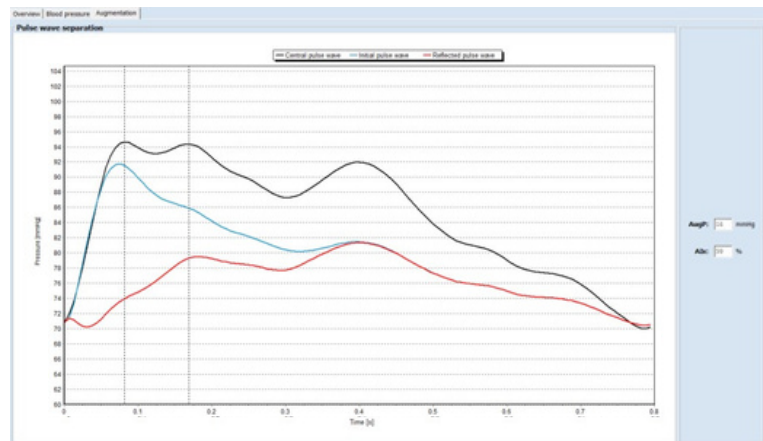


Figure 81: Display of pulse wave separation and augmentation

8 PWA Office

The menu item [PWA Office] is used to perform a single measurement with direct display of the measurement results of a pulse wave analysis in order to obtain a quick statement about the vessel condition in practice. This single measurement is not intended to replace the long-term measurement function, but should be used to justify a long-term measurement.

8.1 Preparation

Start this measurement function by clicking the [PWA Office] button. The patient search is described in section 5.3.1. If the patient is not yet included, a new patient can be created as described in section 5.3.2.



Figure 82: Main menu PWA-Practice

Start the recorder and connect it to the PC according to section 5.1.

Note

Make sure that the selected instrument supports the pulse wave analysis function (see section 3.1).

Click the [Start] button to start a measurement.

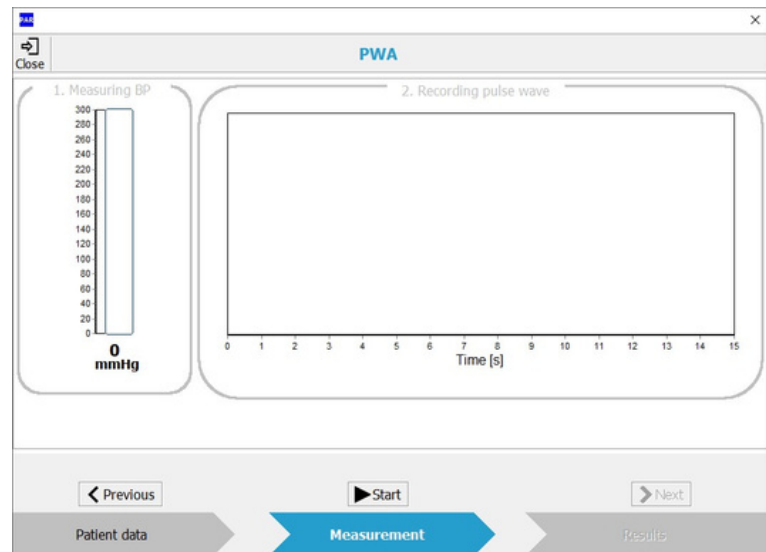


Figure 83: PWA-Pratice

You cannot start measurement when the recorder is turned off or not connected to the PC. In this case, the [Start] button is disabled and the message "Search device" is displayed.



Figure 84: Error indicator "Search Device"

8.2 Description of the measuring procedure

The device starts the pulse wave analysis by default with a blood pressure measurement. In the action area "1. Measuring BP", the current cuff pressure is displayed. After a successful blood pressure measurement and after reaching the determined MAD, the action area is coloured green. In the PWA practice area, different colour codes are used to indicate the status of the measurement to the user. The meaning of the colour code is explained opposite.

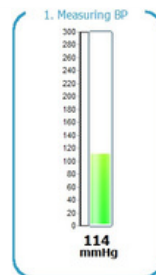


Figure 85: Active measurement

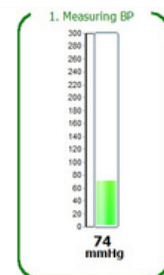


Figure 86: Measurement completed

Gray - Deactive not yet executed

Blue - Active

Green - Successfully completed

Red - Manual abort or incorrect measurement

The measurement can be aborted at any time by clicking the [Stop] button.

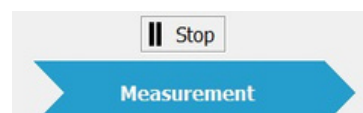


Figure 87: Button to stop measurement

Immediately afterwards, the pulse waves are recorded at the level of the mean arterial pressure. At the end of the recording, the individual pulse waves are displayed separately by their foot points (vertical dashed lines), pulses not used for analysis are marked in red and the signal quality is evaluated based on the number of usable pulses.

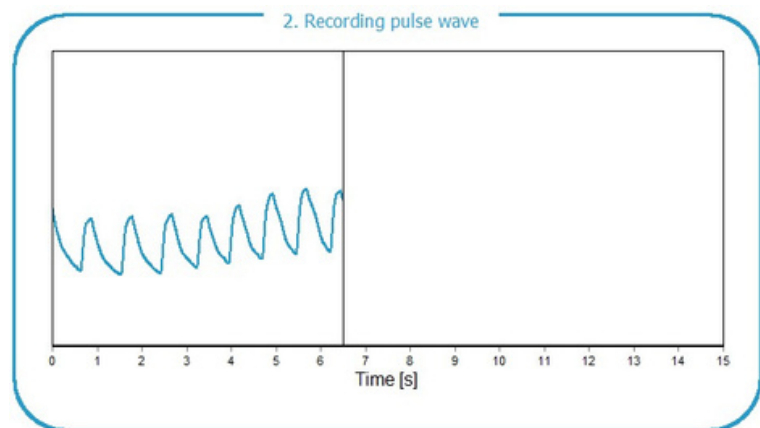


Figure 88: Recording the pulse waves on upper arm

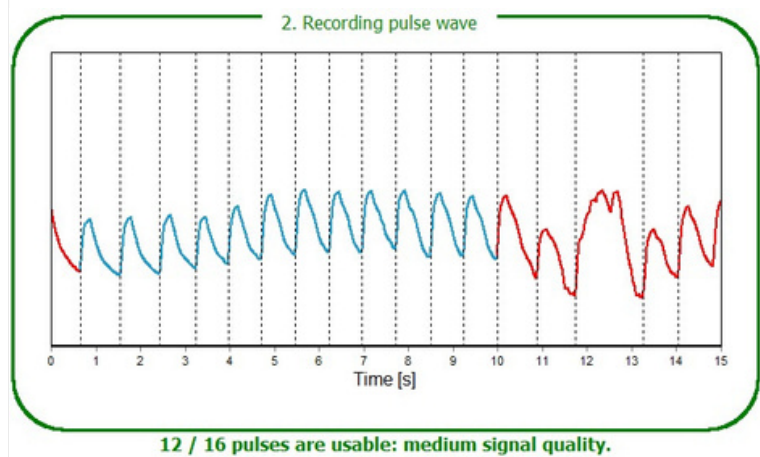


Figure 89: Finished pulse wave recording

By clicking the [Next] button or the [Results] button in the lower navigation area, the results of the pulse wave analysis can be viewed directly. The result area corresponds to the evaluation of a PWA measurement (see section 7.4.2).



Figure 90: Navigation area of the menu item PWA Practice in the "Measurement" section

Note

All successful measurements are automatically stored in the patient database as a single record.

The [Back] or [Measurement] button can be used to return from the result range to the measuring range in order to carry out another new measurement.

Use the [Exit] or [X] button to return to the main menu.



Figure 91: Navigation area of the menu item PWA Practice in the "Results" section

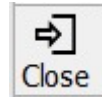


Figure 92: "Exit" button in the PWA Practice menu

9 Additional buttons

9.1 Print

The [Print] button allows you to print the results.

The scope of the printout, i.e. which tabs are to be printed, can be defined individually; a standard format can be predefined under [Settings] (see Section 4.3.3 - Printout).

The integrated print preview, which is activated by selecting "with preview", allows a preliminary evaluation of the created print pages.

The [Print] button starts the printing process.

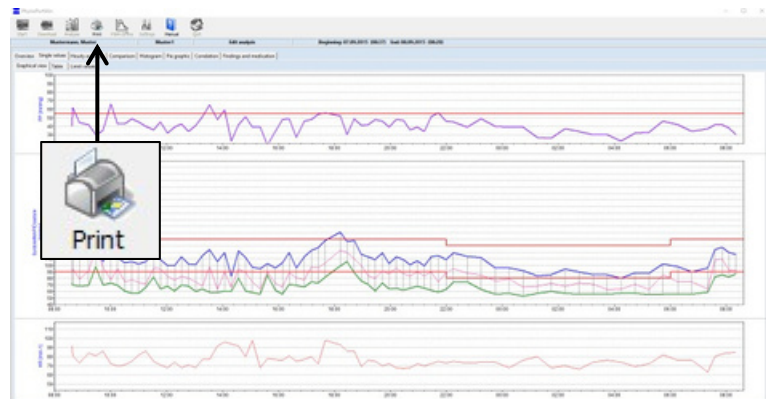


Figure 93: Selection of the menu item "Print"

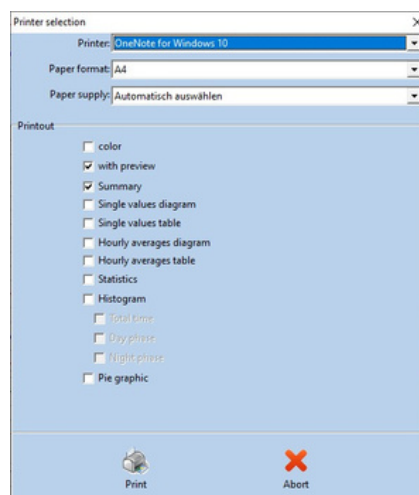


Figure 94: Print dialog

Note

The menu item "Print" can only be selected if a measurement series is open.

9.2 PDF-Export

The [PDF Export] button allows to save the results in PDF format.

The scope of the export, i.e. which tabs are to be exported, can be defined individually; a default format can be predefined under [Settings] (see Section 4.3.3 - PDF Export).

The [PDF Export] button starts the export.



Figure 95: Selection of the menu item "PDF export"

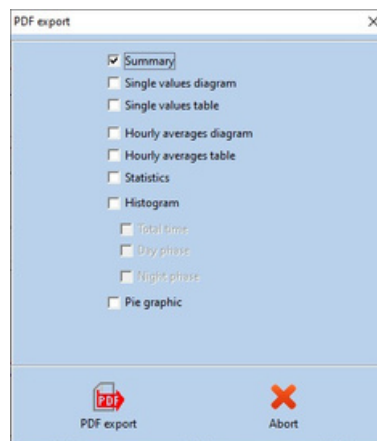


Figure 96: PDF-Export-Dialog

Note

The menu item "PDF export" can only be selected if a measurement series is open.

9.3 GDT - Export to practice-EDP

The [GDT Export] button enables the exchange of patient data and evaluation results between practice/clinic EDP systems and the PhysioPortWin.

This menu is only available if the practice-EDP interface to PhysioPortWin software is activated under [Settings] (see section 4.3.2).

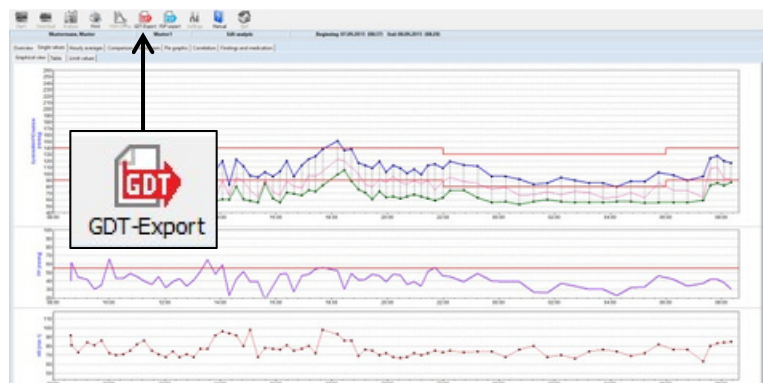


Figure 97: Selection of the menu item "GDT-Export"

With the help of the button [GDT Export], according to the settings made and stored in the configured directory. The successful creation of the GDT file is confirmed by the software via a dialog.



Figure 98: Confirmation of successful GDT export

Note

Only one GDT file can be created at a time for the practice/clinic EDP system. This GDT file is automatically deleted when it has been read by the EDP system. If a GDT file has not yet been read, an error message is displayed.

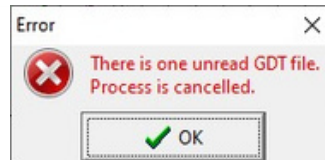


Figure 99: Error message due to an unread GDT file

9.4 CSV-Export

The [CSV Export] button allows you to save the single value table in CSV format and generates a formatted text file with all measured values.

The button can be found on the [Single values] tab with the sub-tab [Table] (see Figure 100), on the [PWA] tab with the sub-tab [Single values] and on the [Overview] tab with the sub-tab [Day/night analysis].

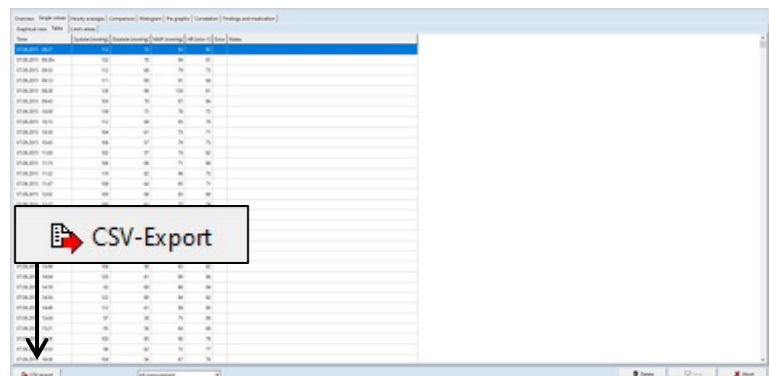


Figure 100: Selection of "CSV export"