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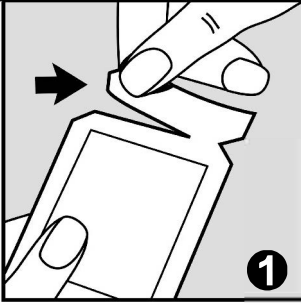
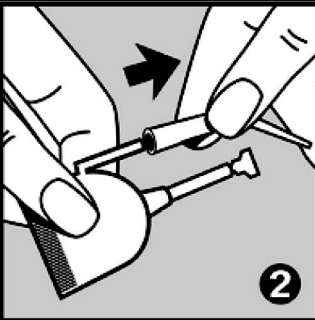
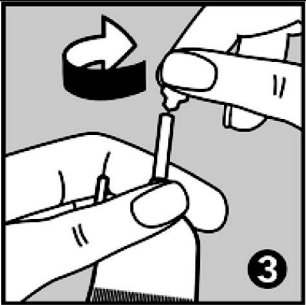
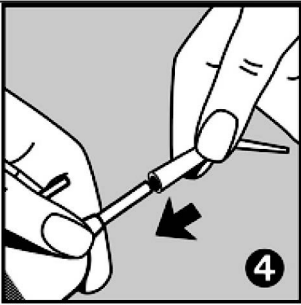
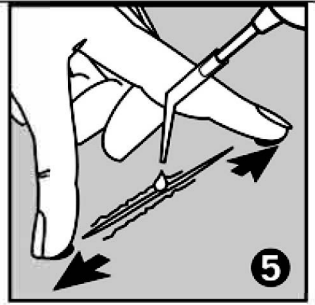
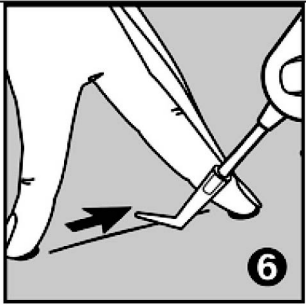
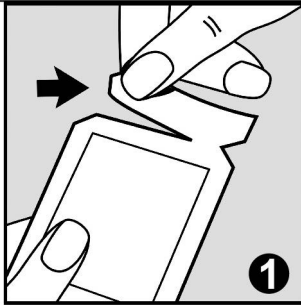
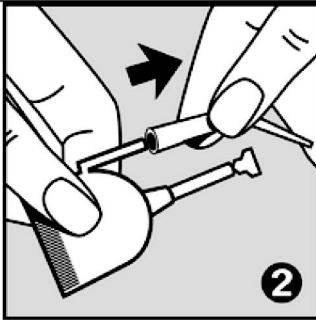
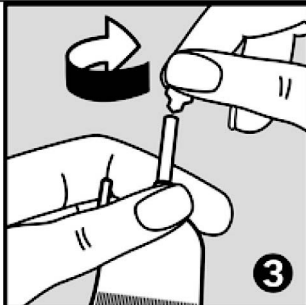
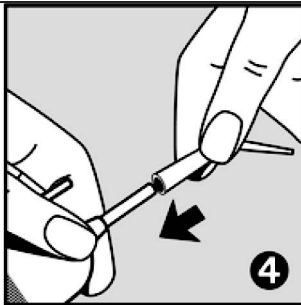
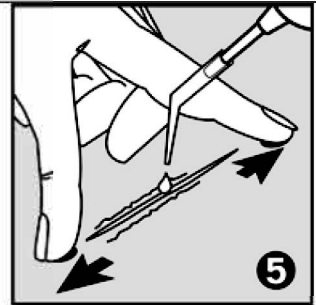
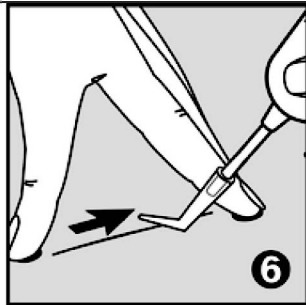
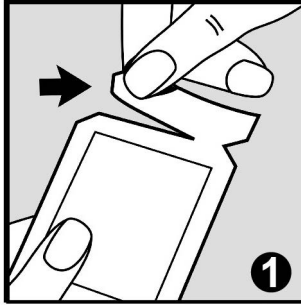
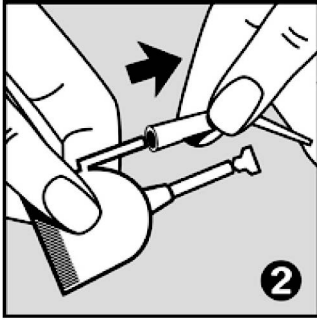
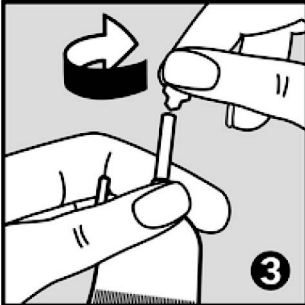
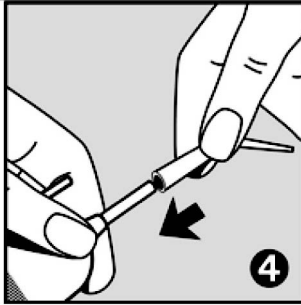
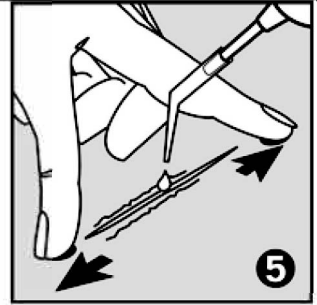
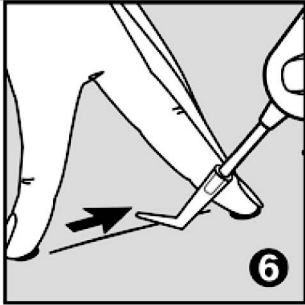
## Schematic diagram 1

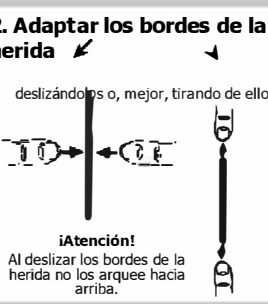
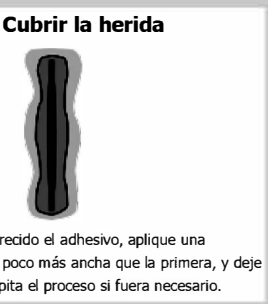
|                                                                                     |                                                                                                                      |                                                                                                                                                                                 |
|-------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|   |                                    |                                                                                               |
| Tear off the aluminium pouch on one of the notches and take out the Dosette.        | Remove the dispenser.                                                                                                | Hold the lower part of the Dosette firmly and twist the upper seal off.<br><b>Important:</b> In order to avoid any leakage whilst opening, do not press the adhesive container. |
|  |                                   |                                                                                              |
| Attach the dispenser to the drip opening of the <b>TRUGLUE®</b> -Dosette.           | Squeeze the reservoir of the Dosette carefully to avoid getting too much adhesive. Drip the adhesive onto the wound. | Smooth the adhesive thinly out on the wound with the bend of the dispenser.<br><b>If required repeat the adhesion process!</b>                                                  |

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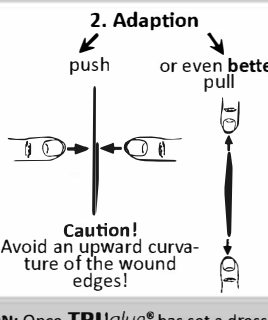
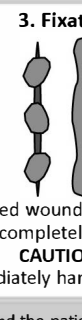
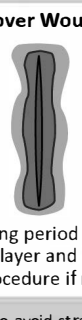
## Schematic diagram 2

### Schematic diagram of a wound treatment with TRUGLUE®

|                                                                                                                                                                                           |                                                                                                             |                                                                                                                                                                      |                                                                                                                                   |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------|
|                                                                                                       |                         |                                                                                  |                                               |
| 1. Dehiscent Wound                                                                                                                                                                        | 2. Adaption<br>push or even better pull<br><b>Caution!</b><br>Avoid an upward curvature of the wound edges! | 3. Fixation<br>Fixate adapted wound edges punctated with TRUGLUE® or completely cover entire wound.<br><b>CAUTION!</b><br>TRUGLUE® immediately hardens on the wound. | 4. Cover Wound<br>After the hardening period apply a second, slightly wider layer and allow to set. Repeat procedure if required. |
| <b>RECOMMENDATION:</b> Once TRUGLUE® has set a dressing may be applied to remind the patient, which is pain free to the greatest possible extent, to avoid straining the wound too early. |                                                                                                             |                                                                                                                                                                      |                                                                                                                                   |

Now the wound is closed secure of infection. The patient is immediately able to resume his personal daily hygiene like washing and showering (not too hot). Please advise the patient not to rub the place treated with TRUGLUE® while foaming and wiping off but only dabbing it carefully. It is recommended not to take a full bath or a sauna.

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#### Notes for health professionals:

|                                                                                                                                                                                                                                                                                                                                                                                            |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>Hazard statements:</b><br/>Causes skin irritation.<br/>Causes serious eye irritation.<br/>May cause respiratory irritation.</p> <p><b>Precautionary statements:</b><br/>Avoid breathing in vapours.<br/>Wear protective gloves / protective clothing / eye protection / face protection.<br/>Wash thoroughly after handling.<br/>Use only outdoors or in a well-ventilated area.</p> | <p>If in contact with eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.</p> <p>Take off contaminated clothing and wash before reuse.</p> <p>If accidentally inhaled:<br/>Move victim to fresh air and keep at rest in a position comfortable for breathing.</p> <p>Keep TRUGLUE® locked up.<br/>Store in a well-ventilated place. Keep container tightly closed.<br/>Disposal of contents / container in accordance with local / regional / national / international regulations.</p> |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

#### Important Information:

- Cyanacrylate: Danger. Bonds skin and eyelids in seconds. Keep out of the reach of children.
- The Safety Data Sheet for TRUGLUE® can be downloaded here:  
<https://www.tshs.eu/en/safety-data-sheets.html>
- Should any serious incidents arise in relation to the medical product TRUGLUE® Single Dose they are to be reported to the manufacturer or the appropriate authorities.

#### To Re-Order

**TRUGLUE®-Single Dose**  
Package with 5 Dosettes each containing 0.3 ml  
Package with 10 Dosettes each containing 0.3 ml  
Package with 25 Dosettes each containing 0.3 ml

|                                                                                                                                                                                                             |                                                                                                                                                             |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Description of the pictograms on the primary and secondary packaging:</b><br><br>Observe instruction for use<br><br>Warning<br><br><b>LOT</b><br>Lot / Batch<br><br>Temperature limit<br><br>Distributor | <p>To be used by: month / year<br/><br/>Do not re-use!</p> <p><b>CE 0482</b> CE-mark and identification number of the Notified Body</p> <p>Manufacturer</p> |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|

#### For further information please contact your local dealer!

|                                                                                                                                                                                                                                                                                               |                                                                                      |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|
| <p>TRUSETAL Verbandstoffwerk GmbH<br/>Konrad-Zuse-Straße 15<br/>33758 Schloß Holte-Stukenbrock<br/>Germany<br/>Phone: +49 5207 991688-0<br/>Fax: +49 5207 991688-28<br/>E-Mail: <a href="mailto:info@tshs.eu">info@tshs.eu</a><br/>Internet: <a href="http://www.tshs.eu">www.tshs.eu</a></p> | <p>MeyerHaake GmbH Medical Innovations, Daimlestr. 4, 61239 Ober-Mörlen, Germany</p> |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------|

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